maxill

PRESTO

INSTANT CHAIRSIDE X-RAY SYSTEM
SYSTÈME RADIOGRAPHIQUE DENTAIRE INSTANTANÉ

FIXER

RTU

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The PRESTO DEVELOPER & FIXER System is formulated to produce x-ray results in approximately one to two minutes.

DIRECTIONS: For instructions on developing, fixing, rinsing and finishing, refer to the instruction chart provided by maxill.

Information: PRESTO FIXER should be used only with PRESTO DEVELOPER.

Warranty: The PRESTO System is guaranteed against manufacturing defects and misformulation.

Storage: Keep container closed when not using. Store in a dry, dark environment at room temperature. Keep away from children. Store away from incompatible products.

Disposal: Environmental regulations vary nationwide. Dispose of used solution in accordance to municipal, provincial and federal regulations in addition to those of your institution.

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PRESTO FIXER

Read Material Safety Data
Sheet Before Using

CAUTION: CAUSES EYE AND SKIN IRRITATION. HARMFUL IF INGESTED. AVOID BREATHING MIST. Avoid contact with eyes and skin. Wear eye/face protection, suitable gloves and other protective clothing when handling. Keep away from heat and oxidizing agents.

EYE CONTACT: Rinse immediately with water. Remove contact lenses if any, and continue rinsing with water for 15 minutes while holding eyelids open. Call a physician immediately.

SKIN CONTACT: Flush skin with plenty of water. Remove contaminated clothing. Call a physician if irritation or burning persists.

INGESTION: Do not induce vomiting. Give large amounts of water. Never give anything by mouth if person is unconscious. Call a physician immediately.

INHALATION: Remove to fresh air. If breathing is difficult, give oxygen. Call a physician immediately.

maxill St. Thomas, ON, Canada, N5R 6C8
1-800-268-8633 www.maxill.com
MADE IN CANADA

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