

maxill u-test[®]

Washer-Disinfecter Cleaning Indicators

Directions for Use – Routine monitoring

Once the correct indicator has been selected using the maxill u-test Evaluation Kit, routine testing can be done to monitor the efficacy of the cleaning process. At the start of each week use 5 indicators to monitor multiple locations in the washer-disinfecter (WD). Testing multiple locations in the same load will monitor the functionality of the spray arm(s) and will help identify instrument spray shadows. For daily monitoring, use a least ONE at the start of each day, ideally ONE indicator in every load.


Storage and Handling

1. Store maxill u-test Ultrasonic Cleaning Indicators in their original packaging between 5 – 30°C at a humidity of 5 – 80% RH.
2. Do not store with other chemicals or in areas with extreme highs and lows of humidity.

Please read all instructions before beginning.

Instructions for weekly 5-indicator test.

1. Following the manufacturer's instructions, prepare the WD to run a load using a verified cleaning cycle.
2. Detach FIVE indicators and place them each in a holder with the indicator side facing out. Take care not to touch the coloured portion of the indicator. Contamination may influence the wash-off characteristics of the indicator.
3. Place FOUR clips around the perimeter of the WD rack in the corners with the indicator facing in. Clip TWO indicators on opposite ends of one shelf and TWO indicators on opposite ends of the other shelf. Clip the remaining indicator in the centre of one of the racks. Alternate the placement of the perimeter indicators and middle indicator weekly.
4. Run the WD cycle. Once the cycle is complete, remove the indicators and check the test area has been cleaned. In the event the entire test area is not removed, refer to the troubleshooting guide.
5. Log the results on the provided maxill u-test Cleaning Indicator Test Log. Once results have been logged the indicators may be discarded.

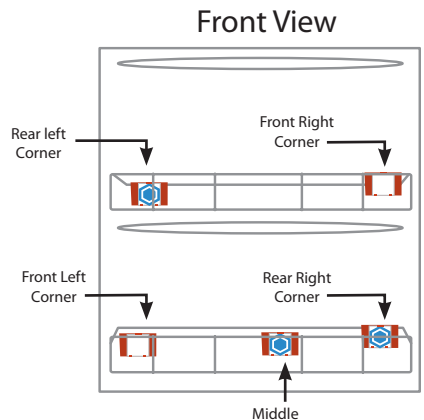


Please read all instructions before beginning.

Instructions for routine testing.

1. Following the manufacturers instructions, prepare the WD to run a load using a verified cleaning cycle.
2. Detach ONE indicator and place it in a holder with the indicator side facing out. Take care not to touch the coloured portion of the indicator. Contamination may influence the wash-off characteristics of the indicator.
3. Attach the holder to the WD rack with the indicator facing into the chamber of the WD. Rotate the position of the indicator around the perimeter of the WD with each test. See the diagram below for the different positions.
4. Run the WD cycle. Once the cycle is complete, remove the indicator and check the test area has been cleaned. In the event the entire test area is not removed, refer to the troubleshooting guide.
5. Log the results on the provided maxill u-test Cleaning Indicator Test Log. Once results have been logged the indicators may be discarded.



Front View

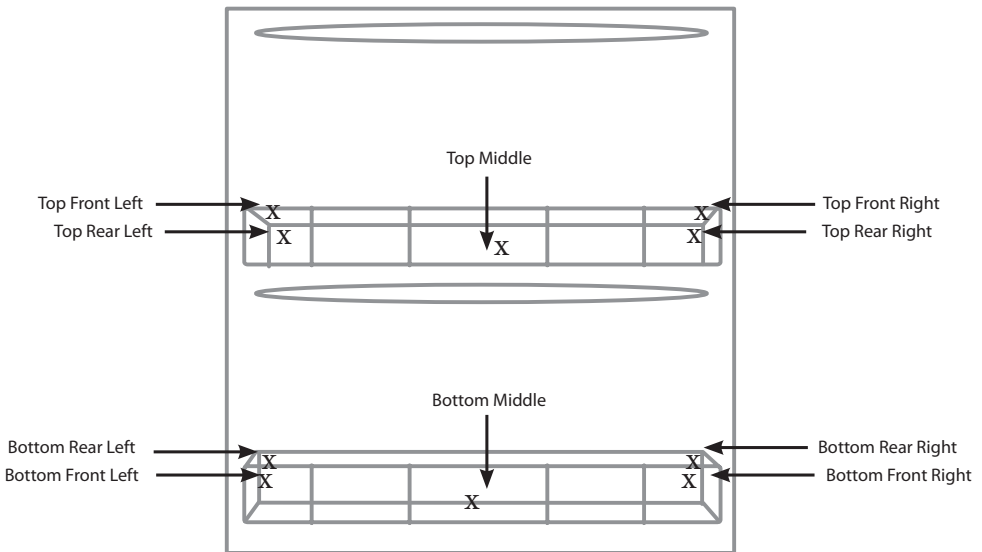

maxill inc.

www.maxill.com 1-800-268-8633

Manufactured for: Manufacturer:
USA: maxill inc. **Canada:** maxill inc.
Cortland, OH St. Thomas, ON
44410 USA N5R 6C8 Canada