

# maxill

ECONOMICAL **64x** CONCENTRATE

## Evaculine 64

DENTAL EVACUATION SYSTEM CLEANER

- Removes organic waste
- Cleans and deodorizes
- Fresh lemon scent
- Corrosion protection
- Non-acid
- Low foaming formulation

When diluted, *Evaculine 64* is designed to clean and deodorize dental institutions' evacuation systems. *Evaculine 64* is a non-acid concentrate to be diluted with water.

**Directions:** Dilute 1 part *Evaculine 64* with 63 parts water, near room temperature.  
E.g. Mix 15 mL / ½ fl. oz. of *Evaculine 64* with water making up a working solution to 0.95 L / 1 qt.  
Aspirate or vacuum the mixed solution through the evacuator lines and cuspidor.  
Repeat this procedure for each dental unit at least once daily.

1 L 33.8 FL. OZ.  
(1.06 QT.)

ITEM 60036

REV.2019/11

## Evaculine 64 - Dental Evacuation System Cleaner

### DANGER

Causes severe skin burns and eye damage.  
May cause an allergic skin reaction.



### PREVENTION

Do not breathe the mist or vapour.  
Contaminated work clothing should not be allowed out of the workplace.  
Wash thoroughly after handling.  
Wear protective gloves/protective clothing/eye protection/face protection.

### RESPONSE

IF ON SKIN (or hair): Take off immediately all contaminated clothing.  
Rinse skin with water.  
Immediately call a POISON CENTER/doctor.  
If skin irritation or rash occurs: Get medical advice/attention.  
Wash contaminated clothing before reuse.  
Specific treatment (see information on this label).  
IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do.  
Continue rinsing.  
IF INHALED: Remove person to fresh air and keep comfortable for breathing.  
IF SWALLOWED: Rinse mouth. Do NOT induce vomiting.

### STORAGE

Store locked up.

### DISPOSAL

Dispose of contents/container in accordance with local/regional/national/international regulations.

Made in Canada with domestic and imported parts

maxill inc.

www.maxill.com 1-800-268-8633

Manufactured for:  
USA: maxill inc.  
Cortland, OH  
44410 USA

Canada: maxill inc.  
St. Thomas, ON  
NSR 6C8  
Canada

REV.2019/11



\*+D89260705600360K\*