


# maxill aquist™ FULL FACE SHIELD

Item **53291**

## Instructions:



### 1 Affix foam strip

Peel the liner from the adhesive of the foam strip and firmly affix the foam to the inside at the top of the shield. (The foam strip may be trimmed to better fit the users head).

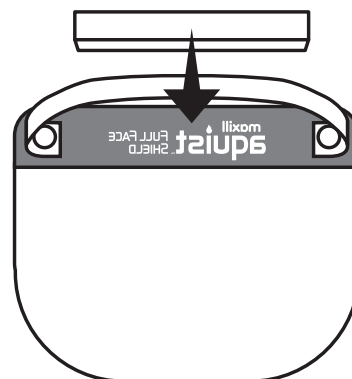

### 2 Remove film

There are protective films on the front and the back of the shield to protect it from from being scratched during transportation. Please remove these films before use.

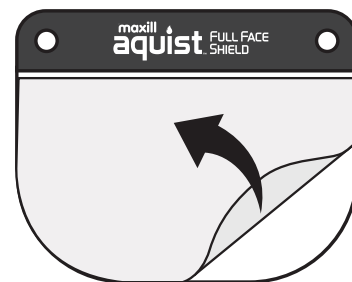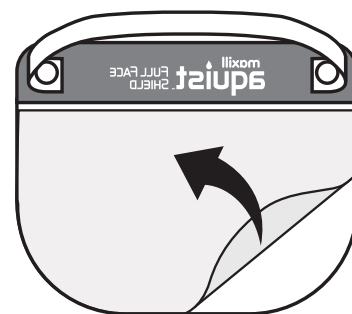

## Warning:

- Use only as a face shield to prevent fluid splashing
- Do not use near heat or open flame
- Do not use to protect against acidic or caustic substances

**maxill inc.**  
www.maxill.com 1-800-268-8633

Manufactured for:  
USA: maxill inc.  
Cortland, OH  
44410 USA

Canada: maxill inc.  
St. Thomas, ON  
N5R 6C8  
Canada

MP-REF-53291D  
Revision 0: 2020-06-16