

# maxill u-test *SI-LR*

## Type 5 Integrating Package Monitoring Indicators

#81125 - 260 per bag  
 #81117 - 500 per bag  
 #81118 - 1,000 per bag  
 #81119 - 2,500 per bag  
 #81121 - 5,000 per bag  
 #81127 - 10,000 per bag



### Handling Information

1. The label can be marked manually with a pen.
2. Place the indicator in the middle of the package or container (most difficult location to sterilize). Alternatively place the indicator unpacked inside the chamber.
3. Run the sterilization program. At the end of the sterilization cycle or after opening of the package, remove the indicator and check the result:
  - If the indicator has changed its colour to the final colour (see label for colour change) the sterilization was successful.
  - If the colour did not change to the final colour, the success of the sterilization process cannot be guaranteed. The contents of the package should **not** be used.
4. All indicators are adhesive and can be documented onto the patient documentation sheet as a record of instrument sterilization.

### Storage and Disposal

1. For longer periods, store all indicators in the original package.
2. Store indicators always between 5-30°C or 41-86°F and a humidity of 5-80% RH.
3. The vapour of chemicals especially hydrogen peroxide may change the indicator before or after sterilization. Therefore, do not store them together with other chemicals.
4. The indicators should not be used after expiry date. They may be disposed of with normal waste.

### Safety Precautions

Monitoring indicators do not replace the basic validation according to national and/or international standards. Follow the instructions of your local requirements.

### WARNING

DO NOT CUT LENGTHWISE TO MAKE TWO STRIPS AS ALTERING THE STRIP NULLS ITS FUNCTION AS A STERILIZATION MONITORING CHEMICAL INDICATOR.



SPALDING'S  
CLASSIFICATIONS

### maxill inc.

**Canadian Head Office**  
 80 Elm Street, St. Thomas, ON N5R 6C8  
 Toll Free: 1-800-268-8633  
 Phone: 1-519-631-7370 Fax: 1-519-631-3388

**USA Head Office**  
 500 West Main Street, Cortland, OH 44410  
 Toll Free: 1-855-462-9455  
 Phone: 1-330-637-1176 Fax: 1-330-637-1154