

[®]maxill

DIAMOND STAINLESS STEEL STRIPS

INSTRUCTIONS FOR USE

INDICATIONS FOR USE:

Finishing and polishing interproximal surfaces via abrasive strips.

INSTRUCTIONS FOR USE:

Abrasive stainless steel sandpaper strips feature a neutral (non-abrasive) center that is used to insert the sandpaper into the proximal faces without wearing down the contact area.

- Hold the sandpaper with the pads of your thumb and index finger
- Insert the files with the neutral center in the interproximals of the teeth
- Make movements from the buccal to the palatal/lingual and vice versa, from the cervical region towards the occlusal surface and vice versa
- In order to facilitate insertion on the proximal face, a lubricant can be used with the sandpaper.

MANUAL CLEANING PROCEDURE:

- Use a designated area specifically for cleaning.
- Rinse the strips under running water and keep them immersed in an enzymatic detergent solution bath, following the manufacturer's instructions.
- After soaking and keeping it soaked, brush well away from your body with a neutral cleaning agent (enzymatic detergent). Care must be taken to avoid spreading contaminants from the spray during brushing procedures. Do not use wire brushes.
- Special care must be taken to clean cracks and other hard-to-reach areas. Visually inspect to confirm debris removal. Repeat the cycle if necessary.
- Rinse the strip well under running water until it is visibly clean.
- Dry the device with clean compressed air or a paper towel.

STERILIZATION REQUIREMENTS:

In case of reuse, it is recommended to sterilize the product in a steam autoclave, properly packed in its own autoclave packaging, or the sterilization parameters must be followed according to the relevant policies/rules in the jurisdiction where the device is being used. It is the full responsibility of the professional to keep the autoclave equipment up to date to ensure the correct sterilization of the product.

CONTRAINDICATIONS:

- Cleaning agents with chlorine or chloride as an active ingredient are corrosive to stainless steel and should not be used.
- Do not use cold sterilization (chemical process) methods to sterilize galvanized steel abrasive strips. These chemical agents can corrode products.

WARNINGS AND PRECAUTIONS:

- Non-sterile product.
- Only dental surgeons should use this product, as they have the scientific information necessary for its proper use.
- Safety glasses must be worn to protect against ejected particles.
- Wear PPE, including gloves, masks, safety glasses, and long-sleeved lab coats when handling the Strip, as with all other dental instruments. The patient should also wear protective eye wear.
- LOT NUMBER: It is important to keep the lot number, identified in the product title, in order to be informed of eventual occurrences with the manufacturer.
- Disposal of material must follow relevant regulations.

STORAGE:

Store in a dry place (cabinet or drawer) in its original packaging or its sterilization pouch.

COMPOSITION:

Stainless steel and diamond.