


FINISHING & POLISHING POLY STRIPS

(100 abrasive strips per box)

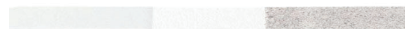


#81435- 2.5mm MEDIUM/FINE



#81436- 4.0mm MEDIUM/FINE



#81437- 6.0mm MEDIUM/FINE

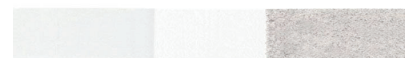

SPAULDING CLASSIFICATION: *Semi-Critical*; Equipment-device that comes in contact with non-intact skin or mucous membranes but does not penetrate them.

INTENDED USE: For use by dental professionals for finishing and polishing interproximal surfaces or stain removal.

MATERIALS: Polyester and aluminum oxide

INSTRUCTION FOR SINGLE USE

Abrasive polyester sandpaper strips feature a neutral (non-abrasive) center that is used to insert the sandpaper into the proximal faces without removing contact point.

- Hold the sandpaper with the pads of your thumb and index finger
- Insert the neutral (non-abrasive) center interproximal between two teeth
- Move the strip in a flossing motion from buccal to lingual/palatal and vice versa, from the cervical region towards the occlusal surface
- Start with the darkest abrasive end (medium coarseness) and end with the lighter abrasive side (Fine Coarseness) until desired finishing and polishing is achieved
- Discard strip after procedure.

WARNINGS and PRECAUTIONS

- non-sterile product
- Do not subject the device to high temperatures
- The use of PPE, including gloves, masks, protective eyewear, and isolation gown should be worn when using during procedures.
- Disposal of material must follow relevant local legislation
- Do not disinfect.
- Do not sterilize.

CONSERVATION and STORAGE

- Store product in a well vented, dry, sterile place, protected from light in order to maintain ideal storage and transport conditions, as well as its physical integrity.
- Keep the product in its original packaging until use. Inspect the package prior to use.
- Dispense/remove number of strips required prior to procedure with clean gloves to reduce risk of cross contamination.
- Packages must not be left exposed to operatory aerosols, or possible contamination.

*Acceptance for North American Healthcare Facilities



Spaulding's Classifications

maxill inc.

Canadian Head Office
80 Elm Street
St. Thomas, ON N5R 6C8
Toll Free: 1-800-268-8633
Phone: 1-519-631-7370
Fax: 519-631-3388

USA Head Office
500 West Main Street
Cortland, OH 44410
Toll Free: 1-855-462-9455
Phone: 1-330-637-1176
Fax: 1-330-637-1154