

Replacement Prophy Rings

(Sold individually)

#53140- Purple Replacement Ring



SPAULDING CLASSIFICATION: Non-Critical Item; Contact intact skin but not mucous membranes. Requires low-level disinfection.

NOTE: A *reusable* non-critical item has the choice of either disinfection or steam sterilization. At minimum a non-critical item must be disinfected.

INTENDED USE: For use by dental professionals in the recommended indications. To hold a single unit-dose cup of prophy paste for ease during prophylaxis procedures.

CLEANING: Reusable non-critical patient care items must be disinfected after use. Clean and disinfect thoroughly between each use with an approved Health Canada or EPA-registered disinfectant.

STEAM STERILIZATION: Autoclavable in Class B, pre-vacuum steam sterilizer up to 275°F (135 °C) for up to 4 mins.

***Acceptance for North American Healthcare Facilities**



Spaulding's Classifications

maxill inc.

Canadian Head Office
80 Elm Street
St. Thomas, ON N5R 6C8
Toll Free: 1-800-268-8633
Phone: 1-519-631-7370
Fax: 519-631-3388

USA Head Office
500 West Main Street
Cortland, OH 44410
Toll Free: 1-855-462-9455
Phone: 1-330-637-1176
Fax: 1-330-637-1154

Rev.1: 2024-04-033

MP-REF-53140D