

**maxill** *xacto stripz*  
**INTERPROXIMAL  
 REDUCTION GAUGES**

#81469

(1 set of 6 gauges)



**SPAULDING CLASSIFICATION:** Semi-Critical item; Contact with mucous membranes or non-intact skin.

**INTENDED USE:** For use by dental professionals. maxill xacto stripz interproximal reduction gauges are used to obtain accurate measurement of the interproximal space resulting from interdental wear (stripping) made with abrasive strips, diamond blades/wheels or any other abrasive device used for interproximal reductions.

**FUNDAMENTALS:** Six stainless steel blades of different thicknesses: 0.05, 0.10, 0.20, 0.30, 0.40, and 0.50 mm, providing total precision and ease.

**INSTRUCTIONS FOR USE:** insert the gauge with the desired thickness to measure the size of the interproximal space. Use in conjunction with interproximal reduction tools for best results. Use in sequential order from smallest to largest gauge until desired interproximal reduction is achieved.

**CLEANING PROCEDURES:**

**Manual Cleaning Only:** Clean with soap and running water using a non-abrasive brush.

**Do not use in ultrasonic or washer/disinfector.**

**Sterilization Requirements:** It is recommended to sterilize the product using a pre-vacuum steam autoclave; maximum temperature of 134°C for up to 10 minutes. Ensure to properly package items according to the policies/rules of the country where the device is being used. It is the sole responsibility of the professional to keep the autoclave equipment updated to ensure proper sterilization of the product.

**Packaging Recommendation:** Fan gauges (see image at top of page) and place type 5 indicator between keys, then place inside sterilization pouch and seal for reprocessing.

**Contraindications**

- Cleaning agents with chlorine or chlorine as the active ingredient are corrosive to stainless steel and should not be used.
- Do not use cold sterilization methods (chemical process) for the sterilization of the gauges. These chemical agents can cause corrosion of the product.



Spaulding's Classifications

**maxill inc.**

**Canadian Head Office**  
 80 Elm Street  
 St. Thomas, ON N5R 6C8  
 Toll Free: 1-800-268-8633  
 Phone: 1-519-631-7370  
 Fax: 519-631-3388

**USA Head Office**  
 500 West Main Street  
 Cortland, OH 44410  
 Toll Free: 1-855-462-9455  
 Phone: 1-330-637-1176  
 Fax: 1-330-637-1154



**maxill** *xacto*  
*stripz*  
INTERPROXIMAL  
REDUCTION GAUGES

#81469



**WARNINGS AND PRECAUTIONS:**

- Only for use by professionals who have been properly trained with the scientific knowledge necessary for its proper use.
- Before use, inspect if the product is in perfect condition of use
- Non-sterile product.
- Sterilize the product before use
- During use professionals must use all personal protective equipment and safety eyewear, including safety eyewear for the patient.
- Used product is considered a contaminated instrument and proper handling precautions are required. Wear PPE including gloves, masks, safety eyewear, and long sleeves when reprocessing.
- The disposal of the material must follow the relevant local legislation.

**CONSERVATION AND STORAGE**

- Product must be stored in an airy, dry place, protected from light, in order to maintain ideal storage and transport conditions, as well as its physical and chemical integrity.
- Keep products in sterile packaging until use. Inspect the packaging before use.

**ADDITIONAL INFORMATION**

- maxill is not responsible if the instructions for use are not followed correctly, generating undesired results in the efficacy and safety of the product or patient.

**MATERIAL:** AISI 304 Stainless Steel

**\*Acceptance for North American Healthcare Facilities**



Spaulding's Classifications

**maxill inc.**

**Canadian Head Office**  
80 Elm Street  
St. Thomas, ON N5R 6C8  
Toll Free: 1-800-268-8633  
Phone: 1-519-631-7370  
Fax: 519-631-3388

**USA Head Office**  
500 West Main Street  
Cortland, OH 44410  
Toll Free: 1-855-462-9455  
Phone: 1-330-637-1176  
Fax: 1-330-637-1154