



## Bur Holder Silicone Holes Flip Lid



- #SCBR0001- 4 Place
- #SCBR0002- 5 Place
- #SCBR0003- 6 Place
- #SCBR0004- 8 Place
- #SCBR0005- 10 Place
- #SCBR0006- 12 Place
- #SCBR0007- 16 Place
- #SCBR0008- 6 Place + Ratchet



**Ohio Forge Bur Holders** are manufactured for use in any cleaning and sterilization process. This item can be used in ultrasonic baths, automatic washers, and autoclaves.

**Material:** 304 stainless steel and medical-grade silicone.

### CLEANING

- After use, wipe and clean the instrument of visible debris before placing it back in the Ohio Forge Bur Holder.
- Clean the bur holder and contained instruments in enzymatic cleaner or place in automatic washer-disinfector using manufacturers recommended solution for automatic unit.
- Ohio Forge Bur Holders are fitted with silicone inserts. These silicone inserts are also suitable for use with any decontamination, cleaning and sterilization process.
- It is important to ensure complete drying before sterilization.

### STERILIZATION

- Sterilize according to AAME/ANSI ST55 and AAMI/ANSI ST8.
- Validate cycle according to AAME/AAMI ST 79 (valid IQ/OQ (commissioning) and product specific performance qualification (PQ)).
- Ensure item is free of debris after the cleaning process.
- Package for sterilization using appropriate size sterilization pouches or surgical wraps.
- Ensure package does not touch other items when placing in sterilizer chamber.
- It is recommended that temperatures do not exceed 360°F (182°C) during any sterilization process.
- It is recommended to follow the manufacturers instructions for reprocessing for the instruments contained by the bur holder.
- Allow to dry completely before removing from sterilizer chamber.



Spaulding's Classifications

#### maxill inc.

**Canadian Head Office**  
80 Elm Street  
St. Thomas, ON N5R 6C8  
Toll Free: 1-800-268-8633  
Phone: 1-519-631-7370  
Fax: 519-631-3388

**USA Head Office**  
500 West Main Street  
Cortland, OH 44410  
Toll Free: 1-855-462-9455  
Phone: 1-330-637-1176  
Fax: 1-330-637-1154