






maxill
aquist 

4-Ply Earloop Style Procedural Masks

maxill aquist 3D EL Earloop Style Procedural Masks are not made with natural rubber latex and offer Level 3 protection*. The diamond-shaped pleats enable easier breathing by creating more room around the nose and mouth, which also helps reduce discomfort. The metal nose piece provides a more secure and personalized fit helping keep the mask comfortably in place.

Availability

- Blue #52991 
 - Pink #52992 
 - Green #52993 
 - Lavender #52994 
 - White #52995 
- (50 masks per box)

***ASTM LEVEL 3**

- Complies with ASTM F2100-19 Requirements:
- Fluid Resistance:** 160 mm Hg
- Bacterial Filtration Efficiency:** BFE ≥ 98%
- Particle Filtration Efficiency:** PFE ≥ 98%
- Breathability - Delta P:** < 5.0 mm H₂O/cm²
- Flame Spread:** Class 1



How to Wear an Earloop Mask



1 Use your fingers to open the ear-loops. Position the nose clip at the top. Place the mask over your nose and mouth.



2 Position the ear-loops around your ears. Pull the mask from the top and bottom to unfold the pleats of the mask.



3 Form the nose clip over the bridge of your nose to make it fit snugly.

Made in Taiwan

The information provided is given in good faith and believed to be accurate, but no warranty or guarantee is given or implied. Conditions of use under which the product is being applied will vary from the conditions of normal laboratory efficacy testing.

MP-REF-52991B
Rev.2: 2025-02-18