



Spaulding's Classification: Non-critical item.

Protective eyewear is considered a non-critical item as there is no contact with mucosa and rests on intact skin. The decision of how to process an item, instrument and/or equipment is based on Spaulding's Classification which states cleaning followed by low-level disinfection. The eyewear is in the vicinity of a high range of aerosols, hence a potential source for cross-contamination, where cleaning alone is not sufficient.

Purpose of use: Protective eyewear in healthcare is to safeguard the eyes of both healthcare workers and patients by preventing exposure to infectious agents, such as bodily fluids, and protecting against chemical splashes from disinfectants and other hazardous substances. Additionally, protective eyewear reduces eye strain from bright lights and instruments and prevents injuries from flying debris or sharp objects, thereby ensuring a safer working environment.

Cleaning:

An intermediate level disinfectant can be used in place of a low-level disinfectant with the realization that the variance in concentrations of the solutions will decrease the longevity of the glasses.

It is recommended post cleaning and disinfection that the eyewear be rinsed with water and dried with a soft lint free towel to remove any build up or smearing left by the disinfectant.

KIDS 265:



- 53265D2- Pink with Black Lenses ●
- 53265D3- Green with Black Lenses ●

ADULT 260:



- 53260- Black with Black Lenses (adjustable arms)

ADULT 267:



- 53267- Black with Clear Lenses

ADULT 275:



- 53275C1- Blue with Clear Lenses ●
- 53275C2- Pink with Clear Lenses ●
- 53275C3- Green with Clear Lenses ●
- 53275C9- Purple with Clear Lenses ●
- 53275C0- Black with Clear Lenses ●

ADULT 276c:



- 53276C1- Blue with Clear Lenses ●
- 53276C2- Pink with Clear Lenses ●
- 53276C3- Green with Clear Lenses ●
- 53276C9- Purple with Clear Lenses ●
- 53276C0- Black with Clear Lenses ●

ADULT 277c:



- 53277C1- Blue with Clear Lenses ●
- 53277C2- Pink with Clear Lenses ●
- 53277C3- Green with Clear Lenses ●
- 53277C9- Purple with Clear Lenses ●
- 53277C0- Black with Clear Lenses ●

ADULT 277d:



- 53277D1- Blue with Black Lenses ●
- 53277D2- Pink with Black Lenses ●
- 53277D3- Green with Black Lenses ●
- 53277D9- Purple with Black Lenses ●
- 53277D0- Black with Black Lenses ●

ADULT 278:



- 53278D1- Blue with Black Lenses ●
- 53278D2- Pink with Black Lenses ●
- 53278D3- Green with Black Lenses ●
- 53278D9- Purple with Black Lenses ●
- 53278D0- Black with Black Lenses ●

ADULT 279:



- 53279D1- Blue with Black Lenses ●
- 53279D2- Pink with Black Lenses ●
- 53279D3- Green with Black Lenses ●
- 53279D9- Purple with Black Lenses ●
- 53279D0- Black with Black Lenses ●

OVERSIZED:



- 53263- 263 Black oversized frames



- 53264- 264 Clear oversized frames



Spaulding's Classifications

maxill inc.

Canadian Head Office
 80 Elm Street
 St. Thomas, ON N5R 6C8
 Toll Free: 1-800-268-8633
 Phone: 1-519-631-7370
 Fax: 519-631-3388

USA Head Office
 500 West Main Street
 Cortland, OH 44410
 Toll Free: 1-855-462-9455
 Phone: 1-330-637-1176
 Fax: 1-330-637-1154