All Purpose Multi-Enzyme Detergent Spray Multi-enzyme

A proprietary blend of enzymes lift and carry away organic material including blood, tissue, and feces

Easy to Use

Naturally reaches hard-to-reach places reducing the need for intensive manual deaning of contaminated surfaces pH Neutral

Will not rust or damage instruments

Directions for cleaning instruments:

- 1. Shake before each use.
- 2. Saturate instruments and equipment with spray gel as soon as possible after the procedure.
- Keep instruments and equipment wet with product until decontamination process is started.
- 4. Rinse with potable water.
- 5. Follow manufacturer's directions for instrument sterilization.

ITEM 60026



REV.2019/07 Made in Canada with domestic and imported parts.

maxill

MULTI-ENZYME RTU DETERGENT



All Purpose Multi-Enzyme Detergent Spray

Multi-enzyme


• pH Neutral Detergent

Zymax Quick Grip Gel is a RTU
multi-enzyme detergent formulated to
be especially effective in the removal of
organic material such as blood, tissue,
mucus, feces, and other organic
material found on instruments.



WARNING

Causes eve irritation and skin irritation.

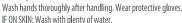

Supplement with hand/body soap if necessary.

Specific Treatment, see First Aid on this label.

If skin irritation occurs, get medical attention.

Take off contaminated clothing and wash it before reuse.

IF IN EYES, Rinse cautiously with water for several minutes.

Remove contact lenses, if present and easy to do. Continue rinsing.

If eve irritation persists: Get medical attention, Refer to SDS.

FIRST AID

Eye Contact: Immediately flush with cool water for at least 15 minutes. Seek medical attention if irritation persists.

Skin Contact: Immediately wash skin with soap and water for at least 20 minutes while removing contaminated clothing. Seek medical attention if irritation persists. Ingestion: Do NOT induce vomiting. Drink cool water. Call a physician or poison control center. Never give anything by mouth to an unconscious person. Inhalation: Move victim to fresh air. Seek medical attention immediately.

STORAGE / DISPOSAL: Store in a cool place. Recycle this container or discard container in trash.



Manufactured for: **USA:** maxill inc. Cortland, OH Canada: maxill inc. St. Thomas, ON N5R 6C8 Canada