

All Purpose Multi-Enzyme Detergent Spray

Multi-enzyme

A proprietary blend of enzymes lift and carry away organic material including blood, tissue, and feces

Easy to Use

Naturally reaches hard-to-reach places reducing the need for intensive manual cleaning of contaminated surfaces

pH Neutral

Will not rust or damage instruments

Directions for cleaning instruments:

1. Shake before each use.
2. Saturate instruments and equipment with spray foam as soon as possible after the procedure.
3. Keep instruments and equipment wet with product until decontamination process is started.
4. Rinse with potable water.
5. Follow manufacturer's directions for instrument sterilization.

ITEM 60025



REV.2019/07 Made in Canada with domestic and imported parts.

maxill

MULTI-ENZYME RTU DETERGENT

Zymax[™] QS
QUICK SHOT ENZYMATIC CLEANING FOAM

All Purpose Multi-Enzyme Detergent Spray

- Multi-enzyme
- pH Neutral Detergent

Zymax Quick Shot Foam is a RTU multi-enzyme detergent formulated to be especially effective in the removal of organic material such as blood, tissue, mucus, feces, and other organic material found on instruments.



WARNING

Causes eye irritation and skin irritation.

Wash hands thoroughly after handling. Wear protective gloves.

IF ON SKIN: Wash with plenty of water.

Supplement with hand/body soap if necessary.

Specific Treatment, see First Aid on this label.

If skin irritation occurs, get medical attention.

Take off contaminated clothing and wash it before reuse.

IF IN EYES, Rinse cautiously with water for several minutes.

Remove contact lenses, if present and easy to do. Continue rinsing.

If eye irritation persists: Get medical attention. Refer to SDS.

FIRST AID

Eye Contact: Immediately flush with cool water for at least 15 minutes.

Seek medical attention if irritation persists.

Skin Contact: Immediately wash skin with soap and water for at least 20 minutes while removing contaminated clothing. Seek medical attention if irritation persists.

Ingestion: Do NOT induce vomiting. Drink cool water. Call a physician or poison control center. Never give anything by mouth to an unconscious person.

Inhalation: Move victim to fresh air. Seek medical attention immediately.

STORAGE / DISPOSAL: Store in a cool place. Recycle this container or discard container in trash.



maxill inc.
www.maxill.com 1-800-268-8633

Manufactured for:
USA: maxill inc.
Cortland, OH
44410 USA

Canada: maxill inc.
St. Thomas, ON
N5R 6C8
Canada

946 mL 32 FL OZ