



KLEARVIEW DISPOSABLE SUCTION MIRROR KIT

PRODUCT DESCRIPTION:

The Klearview system is a single-use, suction-enabled dental mirror designed for effective intraoral visualization and fluid evacuation. This product eliminates the need for separate suction and mirror tools by combining both functions into one ergonomic device. The included adaptors allow quick connection to either high- or low-volume suction systems, offering flexibility across various clinical procedures.

INTENDED USE:

For use by dental professionals to retract tissue, view oral structures, and remove fluids during diagnostic or operative procedures.

CONTRAINDICATIONS & WARNINGS:

- **Do not reuse** any part of the mirror system.
- **Do not autoclave** the mirrors, tubing or adaptors.
- **Do not use** mirror if the sterile pouch is opened or damaged.

DIRECTIONS FOR USE:

Standard Use:

1. Open the sterile pouch containing the mirror using aseptic technique.
2. Choose the appropriate adapter for either high or low suction needs.
3. Connect the mirror to the tubing and suction system.
4. Position mirror intraorally to aid in tissue retraction and fluid control.
5. After the procedure, dispose of the mirror immediately in appropriate waste receptacle.
6. Clean suction lines daily using a turbulence-generating container with an approved evacuation line cleaner.
7. Tubing and adaptors remain on the bracket table just like standard suction tubing and couplings.

Instructions for High Volume Suction Setup:

1. **Insert** the **conversion head** (adapter) into the **tail end of the mirror** (Fig. 1)
2. **Attach** the **tubing** to the **adapter** (Fig. 2)
3. **Connect** the **other end of the tubing** to the **high-volume suction conversion head** (Fig. 3)
4. **Plug** the conversion head into the **high-volume suction port** on the dental unit

STORAGE INSTRUCTIONS:

Temperature range: 0 °C to 35 °C (32 °F to 95 °F)

Relative humidity: 30% to 60%

Protect from sunlight, moisture, and extreme temperature fluctuations.
Store in original packaging

SHELF LIFE:

Mirror sterility is valid until the expiration date printed on the individual pouch. Do not use expired or compromised packages. Prior to the expiration date, mirror sterility is valid unless the pouch is compromised: torn, ripped, punctured, etc.

#SMDK0025

Mirror Kit

- **100 Disposable Mirrors** (assorted colours, individually packaged and sterilized)
- **1 Tubing Segment** (100 cm, flexible)
- **3 Suction Adapters** (for high- and low-volume connection options)



#SMDK0025T


Replacement Tubing (1 tubing segment)

#SMDK0025A

Replacement Adapters (1 adapter set)



Note: The tubing segment and adapters are non-sterile, just like standard dental suction tubing. These components are intended for external handling and must not be sterilized by autoclave.

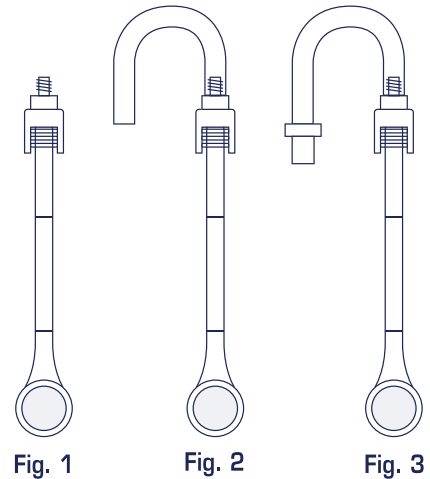

maxill inc.

Canadian Head Office
80 Elm Street, St. Thomas, ON N5R 6C8
Toll Free: 1-800-268-8633
Phone: 1-519-631-7370 Fax: 1-519-631-3388

USA Head Office
500 West Main Street, Cortland, OH 44410
Toll Free: 1-855-462-9455
Phone: 1-330-637-1176 Fax: 1-330-637-1154