

maxill aquist[™] Bouquet

4-PLY EARLOOP STYLE PROCEDURAL MASKS

maxill aquist Bouquet Procedural Masks offer Level 3 protection* in three stylish floral patterns. Made with a premium 4-ply design, these masks feature a soft, non-irritating inner liner and an anti-fog foam strip. aquist Bouquet masks are not made with natural rubber latex.



Lavender Pattern
#53028

Buttercup Pattern
#53027

Leaf Pattern
#53029



Availability

Buttercup Pattern #53027

Lavender Pattern #53028

Leaf Pattern #53029
(50 masks per box)

***ASTM LEVEL 3** Complies with ASTM F2100 Requirements:

Fluid Resistance: 160 mm Hg

Bacterial Filtration Efficiency: BFE ≥ 98%

Particle Filtration Efficiency: PFE ≥ 98%

Breathability - Delta P: < 6.0 mm H₂O/cm²

Flame Spread: Class 1

How to Wear an Earloop Mask



1 Use your fingers to open the ear-loops. Position the nose clip at the top. Place the mask over your nose and mouth.



2 Position the ear-loops around your ears. Pull the mask from the top and bottom to unfold the pleats of the mask.



3 Form the nose clip over the bridge of your nose to make it fit snugly.

Made in China

The information provided is given in good faith and believed to be accurate, but no warranty or guarantee is given or implied. Conditions of use under which the product is being applied will vary from the conditions of normal laboratory efficacy testing.

MP-REF-53024B
Rev.0: 2025-10-09