

maxill
aquist SUREFIT

4-PLY EARLOOP STYLE PROCEDURAL MASKS WITH FLEXIBLE NOSE & CHIN PIECES

maxill aquist SureFit Procedural Masks are designed to provide an enhanced fit. Unlike standard masks which only have a nose piece, aquist SureFit masks feature an additional flexible metal piece along the bottom edge. This unique design allows the mask to fit securely not only across the nose but also under the chin, providing an unparalleled fit.

Constructed with a premium 4-ply design, these masks include a soft, non-irritating inner liner for comfort, as well as an anti-fog foam strip. They are not made with natural rubber latex and provide Level 3 protection*.



Availability

Black #53024
(50 masks per box)

***ASTM LEVEL 3** - Complies with ASTM F2100 Requirements:

- Fluid Resistance:** 160 mm Hg
- Bacterial Filtration Efficiency:** BFE ≥ 98%
- Particle Filtration Efficiency:** PFE ≥ 98%
- Breathability - Delta P:** < 6.0 mm H₂O/cm²
- Flame Spread:** Class 1

How to Wear a SureFit Earloop Mask



1 Use your fingers to open the ear-loops. Position the nose clip at the top. Place the mask over your nose and mouth.



2 Position the ear-loops around your ears. Pull the mask from the top and bottom to unfold the pleats of the mask.



3 Form the nose piece over the bridge of your nose to make it fit snugly.



4 Form the chin piece under your chin to ensure a snug fit.

Made in China

The information provided is given in good faith and believed to be accurate, but no warranty or guarantee is given or implied. Conditions of use under which the product is being applied will vary from the conditions of normal laboratory efficacy testing.

MP-REF-53024B
Rev.0: 2025-10-09