3. Perform the sterilization cycle.

4. After the sterilization process has finished, the dark bar should have reached the ACCEPT zone. If the dark bar has not reached the ACCEPT zone, a REJECT result is indicated and the items in the pouch and/or instrument cassette were not exposed to proper steam sterilization conditions. These items MUST be reprocessed according to instrument reprocessing standards and a new u-test SI-AB chemical indicator must be used. Consult your regional regulatory authorities for guidance on the appropriate steps to take in the event of a failed Type 5 chemical indicator.

NOTE: If a damp appearance is observed on the u-test SI-AB Type 5 chemical indicators after the sterilization cycle, it is recommended to repeat the sterilization cycle. If the problem persists, it is recommended to schedule equipment maintenance.

NOTE: If any serious incident occurs in relation to the device, it should be reported to maxill and the competent authority of the jurisdiction in which the user is established.

Monitoring frequency

Follow dental college and public health regulatory standards/ guidelines for quality assurance sterilization monitoring and logging.

Storage

Indicators should be stored in a dry place, protected from light at temperatures between 10-30 °C (50-86 °F), 30-80 % relative humidity. Do not get wet. Do not store near sterilizing agents.

Shelf Life


Chemical indicators have an expiration date of 5 years from the date of manufacture when stored in recommended conditions. Both the expiration date and batch number of the product are specified on the package label. Do not use indicators after their expiration date.

Endpoint stability reaction: Chemical indicators endpoint shall remain unchanged for a period of not less than 6 months when stored at previously indicated conditions.

Disposal

Discard chemical indicators after use along with paper waste, according to your country's healthcare and safety regulations.

ATCC® is a registered trademark of American Type Culture Collection.

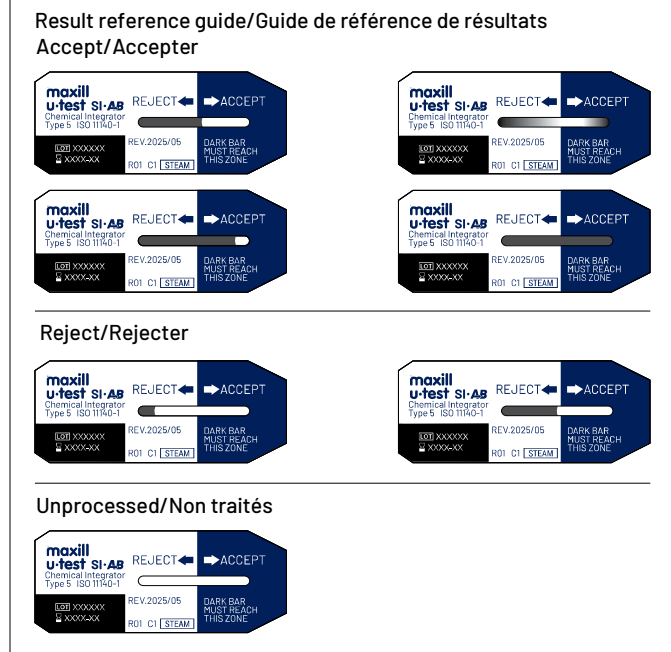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maxill u-test SI-AB Type 5 chemical indicators

Indications for use

maxill u-test SI-AB Type 5 chemical indicators are designed for monitoring steam sterilization processes between 118 °C (244.4 °F) and 138 °C (280.4 °F). See the table below for performance Stated Values (SV) for the integrator.

Model u-test SI-AB (time=minimum exposure duration)		
Steam cycles		
Minimum Stated Values (SV) for Chemical Integrators		
121 °C	128 °C	135 °C
16.5 min.	9.0 min.	1.2 min.

NOTE: The times listed in the table refer to the minimum exposure durations at specific temperatures required for the chemical integrator to indicate successful sterilization—they do not represent the full autoclave cycle time. These values are benchmarks for the integrator's performance, not a substitute for the complete sterilization process. Always follow your sterilizer's manufacturer recommended cycle parameters to ensure proper sterilization and compliance.

Device Description

maxill u-test SI-AB chemical indicators (Type 5 according to ISO 11140-1:2014) are single-use moving front chemical indicators for monitoring steam sterilization processes. These indicators ensure an adequate control of the effectiveness of steam sterilization processes by monitoring all critical parameters. u-test SI-AB Type 5 chemical indicators consist of a special paper wick and a steam and temperature sensitive chemical pellet contained in a paper/film/foil laminate. The chemical pellet melts and migrates as a dark bar along the paper wick. The migration is visible through a zone marked ACCEPT or REJECT, thus indicating whether sterilization conditions were met. The extent of migration depends on steam quality, time and temperature. The ACCEPT result is reached when a theoretical spore population reaches its kill time, indicating integration condition has been reached. This condition is calibrated with the kill time of a 10⁹ *Geobacillus stearothermophilus* ATCC® 7953 spore population, calculated in a BIER (Biological Indicator Evaluator Resistometer).

Precautions

Do not expose u-test SI-AB chemical indicators to ethylene oxide (EO), dry heat, or any sterilization process other than steam. Do not reuse.

Instructions for use

- Carefully open the package and remove only the number of indicators needed. Reseal the package tightly.
- Place one u-test SI-AB Type 5 chemical indicator inside each pouch and/or instrument cassette.

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- Placez un indicateur chimique u-test SI-AB Type 5 à l'intérieur de chaque pochette ou cassette d'instruments.