

DESCRIPTION: **Aqua-FX** is a highly effective and biodegradable disinfectant and fungicide, designed to disinfect plastic water supply lines in medical & dental institutions. When used as directed, **Aqua-FX** will kill vegetative bacteria and thereby reduce the risk of microbial contamination from plastic medical water supply lines constructed of materials such as High Density Polyethylene, Low Density Polyethylene, Polyvinyl Chloride and Polypropylene.

DIRECTIONS FOR USE: Water supply lines should be flushed for 1 to 2 minutes at the beginning of each day and for 30 seconds in between patients.

On a weekly basis, introduce a one-percent solution of **Aqua-FX** (1 part **Aqua-FX** plus 99 parts potable water) and circulate the solution for 5 minutes. To drain the disinfectant from the system, flush the system with potable water for 3 to 5 minutes.

Use low range peroxyacetic acid (PAA) test strips, available from maxill, in order to determine if flushing of **Aqua-FX** from the system is complete. Immerse the test strip in a sample of the rinsate and compare the color of the test strip to the enclosed chart. For technical assistance on using PAA test strips, consult your maxill representative.

Consult your system's manufacturer regarding compatibility of your equipment to **Aqua-FX** (peroxyacetic acid solutions).

[REF] 6070560082



DIN 02332973

Aqua-FX™

**CONCENTRATED DISINFECTANT FOR
PLASTIC MEDICAL WATER SUPPLY LINES**
BACTERICIDE FUNGICIDE

ACTIVE INGREDIENTS:
PEROXYACETIC ACID - 5.1% W/W
HYDROGEN PEROXIDE - 22.7% W/W

INSTITUTIONAL USE ONLY
READ THE LABEL BEFORE USING
FOR EXTERNAL USE ONLY

880 mL

#60082

PRECAUTIONS: Please read the safety data sheet before using. Corrosive concentrated solution! Causes eye and skin damage. Harmful or fatal if swallowed. Do not get in eyes, on skin or on clothing. Wear goggles and rubber gloves when handling. Avoid prolonged breathing of vapors or spray mist. Use in a well ventilated area. **Aqua-FX** is an oxidizing agent. **Aqua-FX** is not combustible, however at temperatures exceeding 68°C, decomposition occurs, releasing oxygen. The released oxygen could initiate or promote combustion of other materials.

FIRST AID TREATMENT: If swallowed, rinse mouth with water. Dilute by giving 1 or 2 glasses of water. Do not induce vomiting. See a physician immediately. If splashed into eyes, flush immediately with water for at least 15 minutes, lifting the upper and lower eyelids intermittently. Get medical help immediately. If splashed on skin immediately flush with water while removing contaminated clothing and/or shoes. Wash skin thoroughly with soap and water. If irritation persists, see a physician. If inhaled, remove to fresh air. If breathing discomfort occurs, see a physician or contact a Poison Control Center immediately.

STORAGE AND DISPOSAL: Keep container closed at all times when not in use. Keep container out of direct sunlight. Store at temperatures between 8°C and 25°C. Triple rinse container with water. Dispose of container in accordance with municipal, provincial and federal regulations. Do not reuse containers.

TOXICOLOGICAL INFORMATION: Note to physician. Probable mucosal damage may contraindicate the use of gastric lavage.

KEEP OUT OF REACH OF CHILDREN

**Made in Canada with domestic and
imported ingredients**

maxill inc.

St. Thomas, ON, Canada N5R 6C8
1-800-268-8633 www.maxill.com

REV.2022/08