Tempit-E

Moisture-Activated Temporary Filling and Sealing Material with Eugenol

GB - ENGLISH INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION: Tempit-E is a viscous, calcium sulphate paste containing eugenol. The material hardens when moisture-activated by water or saliva and expands slightly when completely set. The addition of zinc oxide and barium sulphate provide radiopacity.

INDICATIONS: Short-term temporary fillings, single use only.

DIRECTIONS:

- Prepare site to apply Tempit-E filling material; cavity should be slightly moist.
- Remove one tube from the jar and place into a C-R syringe; remove the cap.
- Squeeze the syringe handle with slow, steady pressure, delivering Tempit-E into the cavity site; dispose after single use.
- Tamp down with moist instrument; wait 5 minutes and trim as needed before dismissing patient.
- On patient's return visit, remove Tempit-E with an explorer.

CAUTIONS & WARNINGS:

- Tempit-E contains a methacrylate resin; do not apply if patient has allergic potential.
- Tempit-E is a temporary filling material; intended to be used for 30 days or less.
- Store unused Tempit-E in supplied jar at all times to prevent premature hardening of material.
- Dispose of Tempit-E properly according to local and national requirements.
- Do not use Tempit-E beyond expiration date stated on container.

DISCLAIMER: Centrix products have been developed for use in dentistry. We have provided written, verbal and online information as to the use of each product. This information does not relieve you from examining our products upon receipt to determine their suitability for use in the intended manner of application. The usage of our products is beyond our control and the user maintains full responsibility. We guarantee the quality of our products in accordance with the existing standards and corresponding to the conditions stated in our instructions for use and safety data sheets.

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Inhalation: Irritating to respiratory system. May cause coughing, tightness of the chest. Move exposed person to fresh air. Seek medical attention.
Eyes: Flush with copious amounts of water for 15 minutes with eyes wide open. If irritation persists seek medical attention.

SECTION V – FIRE-FIGHTING MEASURES

Burning produces irritating, toxic and obnoxious fumes. Suitable extinguishing.
Media: Use extinguisher suitable to cause of fire and its surroundings.
Special protection equipment: Wear suitable respiratory equipment when necessary.

SECTION VI – ACCIDENTAL RELEASE MEASURES

Avoid raising dust. Ensure adequate ventilation of the working area. Wear suitable protective equipment.

SECTION VII – HANDLING AND STORAGE

Handling: No handling problems are likely to be encountered provided normal safe practices of hygiene are applied.
Storage: Store in a cool dry place
Containers: Keep containers tightly closed. Store in correctly labelled containers.
Incompatibilities: Unknown

SECTION VIII – EXPOSURE CONTROLS / PERSONAL PROTECTION

General measures: Wash hands thoroughly after contact with this product.
Limits: No data is available
Respiratory Protection: Wear suitable respiratory equipment when necessary
Eye protection: Goggles / eyeshields
Skin protection: Overalls, rubber gloves
Hands protection: Overalls, rubber gloves
Other equipment: No further information available

 centrix® MAKING DENTISTRY EASIER®	
 CENTRIX INCORPORATED 770 River Road, Shelton, CT 06484 USA (203) 929-5582 (800) 235-5862 FAX (203) 929-6804 www.centrixdental.com	
 CE 0482 Dental-Contact Vertriebs OHG, Im Spiet 101, DE 26506 Norden	
 EMERGO AUSTRALIA Level 20, Tower II, Darling Park, 201 Sussex Street Sydney, NSW 2000, Australia	