

maxill Plus

EARLOOP STYLE PROCEDURAL MASKS WITH ANTI-FOG VISOR

maxill Plus Earloop Style Procedural Masks with Visor are not made with natural rubber latex and offer Level 3 protection*.

*ASTM LEVEL 3

Complies with ASTM F2100-19 Requirements:

Fluid Resistance: 160 mm Hg

Bacterial Filtration Efficiency: BFE ≥ 98%

Particle Filtration Efficiency: PFE ≥ 98%

Breathability - Delta P: < 5.0 mm H₂O/cm²

Flame Spread: Class 1

Availability

| | |
|-------------|--------|
| Blue | #53041 |
| Pink | #53042 |
| Green | #53043 |
| White | #53045 |

(25 masks per box)



How to Wear an Earloop Mask



1 Use your fingers to open the ear-loops. Position the nose clip at the top. Place the mask over your nose and mouth.



2 Position the ear-loops around your ears. Pull the mask from the top and bottom to unfold the pleats of the mask.



3 Form the nose clip over the bridge of your nose to make it fit snugly.

Made in China

The information provided is given in good faith and believed to be accurate, but no warranty or guarantee is given or implied. Conditions of use under which the product is being applied will vary from the conditions of normal laboratory efficacy testing.

MP-REF-53041B
Rev.0: 2020-08-10