

maxill

ITEM 61500

Cleanse-a-dent

Denture Cleanser

Directions: Remove and rinse dentures in running water. To make a measuring cup dilution, take 45 mL (3 tbsp) of maxill Cleanse-a-dent and add tap water to make 1 cup (237 mL). Allow the dentures to soak for approximately 10 minutes. Afterwards brush and rinse the dentures very well under running tap water. For stubborn stains, soak the dentures for a longer period of time. Prepare a fresh solution when solution becomes visibly dirty.

Precautions: Do not use Cleanse-a-dent Denture Cleanser directly in the mouth. Wash hands thoroughly after handling. Keep out of reach of children.

First Aid: If on skin, wash with plenty of soap and water. If in eyes, rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If skin or eye irritation persists: Get medical attention.
Made In Canada with domestic and imported parts.

Détergent pour Dentier

Mode d'emploi: Retirer et rincer les dentiers avec l'eau. Pour faire une tasse à mesurer diluée, prendre 45 mL (3 cuillères à soupe) de maxill Cleanse-a-dent et ajouter l'eau du robinet pour faire 237 mL (tasse à mesurer). Permettre les dentiers d'être imbiber environ de 10 minutes. Puis brosser les dentiers sous l'eau courante. Pour des taches très difficile, imbiber les dentiers pour une période plus longue. Préparer une nouvelle solution quand le liquide devient visiblement sale.

Précautions: Ne pas ingérer. Rincer soigneusement les mains après manipulation. Garder hors de la portée des enfants.

Premiers Soins: PEAU: Laver abondamment au savon et à l'eau. YEUX: Rincer avec précaution à l'eau pendant plusieurs minutes. Enlever les lentilles de contact si la victime en porte et si elles peuvent être facilement enlevées. Continuer à rincer. Si l'irritation des yeux persiste ou l'irritation cutanée persiste: demander un avis médical.

Fait au Canada avec des composants canadiens et importés.

maxill inc.

www.maxill.com 1-800-268-8633

Manufactured for: Manufacturer:
USA: maxill inc. Canada: maxill inc.
Cortland, OH St. Thomas, ON
44410 USA N5R 6C8 Canada

4 L • 1.06 GAL

REF 61500 REV.2017/02

