

Cleaning Instructions for **aquist mini-shields**

(Item 53280)



Cleaning

The aquist Mini Shield reusable frames can be washed with lukewarm water and a non-abrasive, mild soap. Ensure to rinse thoroughly with water after washing to remove any soap residue. Dry completely before disinfection.

Disinfection

The Aquist Mini Shield has reusable frames that can be disinfected with a broad spectrum disinfectant observing the kill times of the disinfectant. Any disinfection process must be preceded by proper cleaning.

NOTE: Only the frame can be reused. The plastic shields are disposable. Applying Spaulding's Classification the reusable frames are a non-critical item (do not come into contact with oral mucosa and are not used subgingival) and can be reprocessed via disinfection.