

Best Practices In Steam Sterilization

maxill

Gravity Displacement Steam Sterilizer

Tutt-nauer - 2340M, 2540M, 2540MK, 3870M, Valueklave - 1730

Dynamic Air Removal For Steam Flush Pressure Pulse (SFPP)

Tutt-nauer - EZ9, EZ10, EZ10K, 3870EA, Midmark - M3, M9, M11, Scican Statim - 2000, 2000 G4, 5000, 5000 G4

 **BI + TYPE 5 IN A PCD & CONTROL DAILY** (Loads containing implantable devices must also be monitored with a BI and quarantined)



u-test PCD
Process Challenge Device
ITEM #81155 - #81158



u-test BI
Biological Indicators
ITEM #81150



u-test
Incubator
ITEM #81120

 **TYPE 5 INTEGRATOR IN EVERY LOAD (BATCH MONITORING)**
(Best practice includes a type 5 in every cassette and package.
Type 5 in every package may vary depending on geographical location)



u-test SI LR
Integrating Package
Monitoring Indicators
ITEM #81125

 **POUCH with external type 1 indicator & internal type 4**
(Use a type 5 indicator in every pouch if required by local regulations)



steri-SOX™
Sterilization Pouches
10 SIZES TO CHOOSE FROM (see catalogue for sizes)

Or

WRAP with external type 1 indicator & internal type 5
(Use with maxill Autoclave Tape and Type 5 Integrators)



SAFEGUARD
Sterilization Wrap
SMMS & PAPER (see catalogue for sizes)

 **LABEL your package for easy identification** (Labels are pre-printed with the required fields to be affixed prior to sterilization. Labels are heat tolerant.)

DATE
2018/06/14
CONTENTS
PERIO1
UNIT # LOAD #
7586 154

Label Sample
(blue text is pre-printed)



steri-SOX™
Quick Label System
LABELER - ITEM #91140
LABELS - #91141

10/05/2019
↑ DATE ↓ CONTENTS
PERIO
UNIT# LOAD#
123 789

Label Sample
(blue text is pre-printed)



steri-sox ID+™
Quick Label System
LABELER - ITEM #91145
LABELS - #91146

MP-REF-81121D
Rev.1: 2019-04-10

www.maxill.com

1-800-268-8633

Best Practices In Steam Sterilization

maxill

Dynamic Air Removal Pre-Vacuum (Class B)

Scican Bravo, Adec Lisa, Tuttnauer Elara 11

 **AUTOClave WARM-UP CYCLE DAILY** (Complete a warm-up cycle with an empty chamber at the start of each day)

 **TYPE II BOWIE-DICK TEST DAILY**
(Test daily after warm-up cycle in an empty chamber)



u-test BDS
Bowie-Dick Simulation Test
ITEM #81190
Bowie-Dick Simulation Test Strips
(TYPE II)
ITEM #81191

 **BI + TYPE 5 IN A PCD & CONTROL DAILY** (Loads containing implantable devices must also be monitored with a BI and quarantined)



u-test PCD
Process Challenge Device
ITEM #81155 - #81158



u-test BI
Biological Indicators
ITEM #81150



u-test
Incubator
ITEM #81120

 **TYPE 5 INTEGRATOR IN EVERY LOAD (BATCH MONITORING)**
(Best practice includes a type 5 in every cassette and package.
Type 5 in every package may vary depending on geographical location)



u-test SI LR
Integrating Package
Monitoring Indicators
ITEM #81125

 **POUCH with external type 1 indicator & internal type 4**
(Use a type 5 indicator in every pouch if required by local regulations)

WRAP with external type 1 indicator & internal type 5
(Use with maxill Autoclave Tape and Type 5 Integrators)




steri-sox™
Sterilization Pouches
10 SIZES TO CHOOSE FROM (see catalogue for sizes)

Or



SAFEGUARD
Sterilization Wrap
SMMS & PAPER (see catalogue for sizes)

 **LABEL your package for easy identification** (Labels are pre-printed with the required fields to be affixed prior to sterilization. Labels are heat tolerant.)

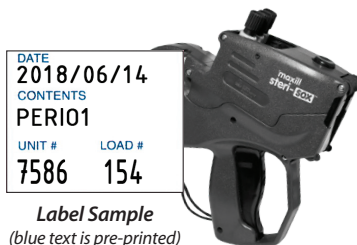

steri-sox™
Quick Label System
LABELER - ITEM #91140
LABELS - #91141



steri-sox ID+™
Quick Label System
LABELER - ITEM #91145
LABELS - #91146