maxill u-test SI-UR

> Type 5 Integrating

Package Monitoring Indicator ITEM 81125

EN ISO 11140-1

STEAM

L PASS





U-testsur

Integrating Package Monitoring Indicators

maxill u-test SI LRs are Type 5 chemical integrators used to monitor the steam sterilization process. These indicators provide reliable quality assurance by measuring all three parameters of sterilization.

The integrating indicator reacts to time, temperature, and pressure with a change in colour. The easily interpreted results indicate a pass when the colour bar undergoes a full colour change, from yellow to black.

The SI-LR meets the EN ISO 11140-1 standard and comes label ready with a convenient adhesive backing for easy record keeping.

- ► Monitors all 3 critical parameters of steam sterilization
- **Easily interpreted results with** precise colour change
- ► Compact design saves space and reduces waste
- Label ready backing for convenient record keeping



EN ISO 11140-1 **STEAM**

Product Availability

Integrating Monitoring Indicators#81125 (260 integrators per bag)

Made in Germany

The information provided is given in good faith and believed to be accurate, but no warranty or guarantee is given or implied. Conditions of use under which the product is being applied will vary from the conditions of normal laboratory efficacy testing.

MP-REF-81125B Rev. 0: 2018-09-13