


Cleaning and Sterilization Instructions for **Autoclavable Glass Dappen Dishes**

Items 53113, 53114, 53115

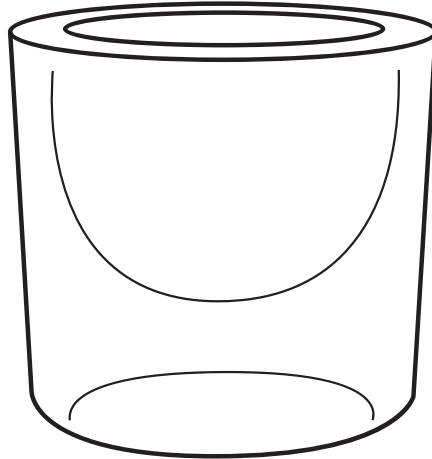

Before and after use glass dappen dishes must follow a cleaning procedure and proper sterilization.

Cleaning

Glass dappen dishes should be cleaned with an enzymatic cleaner in an ultrasonic bath. Rinse with running water, dry completely, package and label for sterilization. An instrument washer can be used for the cleaning process. Ensure dappen dishes are thoroughly rinsed and fully dried prior to packaging for sterilization. Improper rinsing can leave a chemical residue from the cleaning agent and cause a reaction with the high heat from sterilization.

Sterilization

Glass dappen dishes can be heat sterilized in an autoclave sterilizer up to 135°C (275°F). After sterilization, dappen dishes are to remain in their sealed package and opened only at point of use.