



DOVE®

maxill

Disposable Valves Tailpiece Options

HVE and SE Valve Tailpieces are designed to securely connect to the HVE or SE tubing to accommodate DOVE Disposable Valves. These tailpieces are simple to install and are compatible with dental chair hangers from all of the most popular manufacturers.




Step 1




Locate the HVE or SE tailpiece to be removed at the end of the vacuum tubing.

Step 2

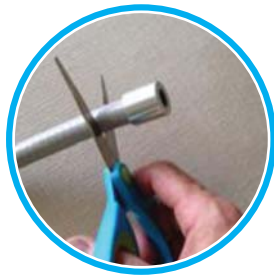

Cut off the existing tailpiece, approximately two inches below its current position.

Step 3

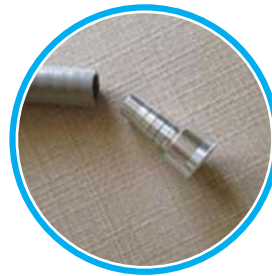

Push the new DOVE® tailpiece on the existing vacuum tubing, using dish soap or natural lubricant as needed.

Step 4



Install and use the disposable DOVE® HVE or SE valve.

Standard HVE Tailpiece

Item 53085HS

\$20.75 each



Recommended for the following Dental Chair Hangers:




Wide HVE Tailpiece

Item 53085HW

\$20.75 each



Recommended for the following Dental Chair Hangers:

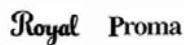

SE Valve Tailpiece

Item 53084S

\$18.25 each



DOVE SE Valves fit most manufacturers tailpiece adapters sold in North America. We offer the SE Valve Tailpiece to accommodate unique fits.