


# maxill lumax

ITEM 91210

## CURING LIGHT METER

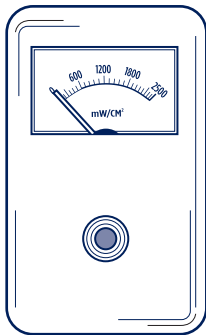

### OPERATION:

To get a correct reading, the light guide must be held flat against the light measuring circle. When the light guide is in the proper position, switch on the curing light. The output of the light will register on the meter and can be easily read (no batteries are required for operation).

### CLEANING AND DISINFECTION:

Light meter can be wiped with non-alcohol based disinfectant wipes.

### SPECIFICATIONS:

Wavelength Range: 440-480 nm  
Measurement Range: Up to 2500 mW/cm<sup>2</sup>

**THIS CURING LIGHT METER IS INTENDED FOR PROFESSIONAL DENTAL APPLICATION ONLY.**