

# maxill *Silken*

## EARLOOP STYLE PROCEDURAL MASKS

**maxill Silken Earloop Procedural Masks** are not made from natural rubber latex and offer Level 3 protection\*. The anti-fog foam strip provides added comfort and helps keep your glasses fog free.

**\*ASTM LEVEL 3**

Complies with ASTM F2100-19 Requirements:

**Fluid Resistance:** 160 mm Hg

**Bacterial Filtration Efficiency:** BFE ≥ 98%

**Particle Filtration Efficiency:** PFE ≥ 98%

**Breathability - Delta P:** < 5.0 mm H<sub>2</sub>O/cm<sup>2</sup>

**Flame Spread:** Class 1

**Availability**

Blue.....	53061
Pink.....	53062
Green.....	53063
White.....	53064

(50 masks per box)



### How to Wear an Earloop Mask



**1** Use your fingers to open the ear-loops. Position the nose clip at the top. Place the mask over your nose and mouth.



**2** Position the ear-loops around your ears. Pull the mask from the top and bottom to unfold the pleats of the mask.



**3** Form the nose clip over the bridge of your nose to make it fit snugly.

Made in China

The information provided is given in good faith and believed to be accurate, but no warranty or guarantee is given or implied. Conditions of use under which the product is being applied will vary from the conditions of normal laboratory efficacy testing.

MP-REF-53061B  
Rev.0: 2020-11-02