

maxill *Silken*^{4P}

4-PLY EARLOOP STYLE PROCEDURAL MASKS WITH ANTI-FOG VISOR

maxill Silken 4P Earloop Procedural Masks are not made from natural rubber latex and offer Level 3 protection*. The anti-fog foam strip provides added comfort and helps keep your glasses and the visor fog-free. The wrap-around anti-fog visor provides additional built-in protection against spray and splatter.

*ASTM LEVEL 3

Complies with ASTM F2100-19 Requirements:

Fluid Resistance: 160 mm Hg

Bacterial Filtration Efficiency: BFE ≥ 98%

Particle Filtration Efficiency: PFE ≥ 98%

Breathability - Delta P: < 5.0 mm H₂O/cm²

Flame Spread: Class 1

Availability

Blue..... 52980

(25 masks per box, 10 boxes per case)



How to Wear an Earloop Mask



1 Use your fingers to open the ear-loops. Position the nose clip at the top. Place the mask over your nose and mouth.



2 Position the ear-loops around your ears. Pull the mask from the top and bottom to unfold the pleats of the mask.



3 Form the nose clip over the bridge of your nose to make it fit snugly.