

Nuer Gauze™ Dental Dressing

Product Specifications

Description

Nuer Gauze Dental Dressing is made from biocompatible regenerated cellulose, chemically treated to become water-soluble, and contains no chemical additives. Upon contact with a moist oral wound, the material dissolves and transforms into a gelatinous material. By applying gentle pressure at this time, the material will adhere to the wound and form a barrier, protecting the wound from further irritation. Nuer Gauze Dental Dressing dissolves in a few days and is safe if swallowed. Excess dressing material may be removed by rinsing with sterile water or saline solution.

Indications

Nuer Gauze Dental Dressing is intended for use as a wound dressing in extraction sites, and may be used as a wound dressing for the temporary management of oral surgical wounds, such as operative, postoperative and traumatic injuries.

Contraindications

Patients with allergies to the sodium carboxymethylcellulose (CMC) should refrain from using this product.

Warning

Nuer Gauze Dental Dressing can not be implanted. The dressing material should be rinsed away with sterile water or saline solution before suturing.

Caution

Federal US law restricts this device to sale by or on the order of a physician.

Read the instructions before use.

Make sure gloves and devices keep dry before use.

The contents of each blister cavity are sterile unless opened or damaged. Discard unused portion immediately after use.

The dressing should not be used on dirty, contaminated or infected extraction sockets.

Consult your doctor if you notice any itching, burning, rash or signs of infection such as fever, pain, redness or swelling.

This product is not reusable. Single-use only.

Instructions for Use

1. Cut or fold Nuer Gauze Dental Dressing as needed to cover the wounded area.
2. Gently apply to the wounded site for product activation.
3. Repeat the steps above with another piece of Nuer Gauze as needed.
4. Excess dressing material may be removed by rinsing with sterile water or saline solution. If allergy occurs, stop using Nuer Gauze Dental Dressing or contact a doctor/medical help.

Packages and Storage

Nuer Gauze is available sterile-packed individually.

Keep Nuer Gauze in a cool dry place for maximum shelf life. The preferred storage temperature should be between 5°C and 40°C.

Sterile

Nuer Gauze is sterilized in the original packaging by gamma radiation.

Shelf Life

Three years.