# maxill

## TOUCH-UP TOOTH WHITENING PEN

### Instructions:

- 1) Brush and floss your teeth before each application.
- 2) Each pen contains 2.5 mL of premium whitening gel which represents approximately 14 applications.
- 3) Remove cap from pen to expose the brush. Turn pen end to dispense gel into the brush. Avoid using too much gel. Brush a thin layer of gel on the front surface of your teeth.
- 4) Place the cap back onto the pen. This protects the gel from exposure and maintains shelf life.
- 5) Keep your mouth open and lips away from gel for 2-5 minutes to allow the gel to take effect. Application times are only suggested and approximate, depending on your comfort, available time frame and individual sensitivity issues.
- 6) Upon completion of each application, rinse or brush your teeth to remove any excess whitening gel.
- 7) Continue using this system uninterrupted once or twice a day until all of your applications have been completed or the desired results are achieved.

### Other Considerations:

- Wait at least one hour between applications.
- Store whitening pen in a cool, dry place. Refrigerate if possible.
- Avoid citrus fruits and juices until your applications have been completed, since they may cause sensitivity.
- Reduce or eliminate consumption of coffee, tea, red wine, cola and tobacco, since they may contribute to tooth staining during (and after) your whitening application.
- Brush and floss before and after each whitening application.

### Warning:

If irritation (such as redness, swelling, soreness) of the gums or the mouth occurs, discontinue use and consult a dentist. Products containing peroxides are not recommended for use by children under 12 years of age. The use for periods of longer than 14 days is to be only under the supervision of a dentist. Avoid swallowing the gel or part thereof. Avoid contact of the product with the eyes. Avoid direct contact of the active surface of the tooth whitening product with the gums and/or salivary flow. Do not use if you are pregnant, lactating or think you may be pregnant.

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