

maxill maxsèt



Item 53776

TYPE I: Fast setting compliant with ISO 21563 specification.

maxill maxsèt does not require special treatment before use.

PURPOSE: maxill maxsèt is a non-chromatic alginate with long dimensional stability specifically designed with the same optimal application specifications as maxill chrömax. The orange colour allows an excellent contrast and tool to evaluate the quality of the impression.

maxill maxsèt alginate can be used for the preparation of:

- Preliminary models
- Study models
- Mobile prosthesis
- Removable and fixed prosthesis antagonist

INSTRUCTIONS:

Follow the powder/water ratio. Pour the powder into the water, and mix vigorously. Work in compliance with the declared times and chromatic phases. Measurements were taken at 23°C with deionized water. When using specific mechanical mixers for alginate-based impression materials, follow the manufacturer's instructions. Variations in powder/water ratio, temperature and/or water hardness, and speed and/or friction during mixing can cause changes in the working time and the initial setting time.

WARNINGS: For use, wear face mask, protective gloves, eye protection, and protective clothing. The powder is "Dust Free", however, caution is recommended to avoid inhalation. In case of contact with eyes wash thoroughly with water. After using the powder, close the bag immediately. Store in a cool and dry place. Use within the expiry date. Do not use on patients allergic to berry flavours.



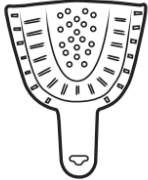



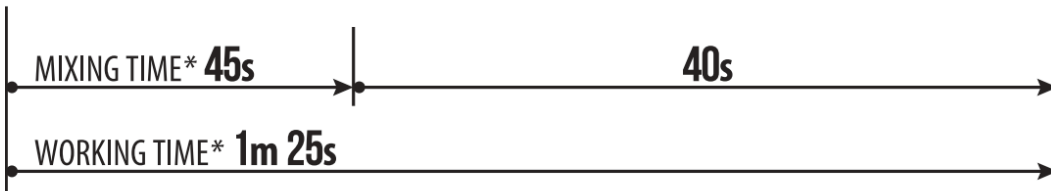
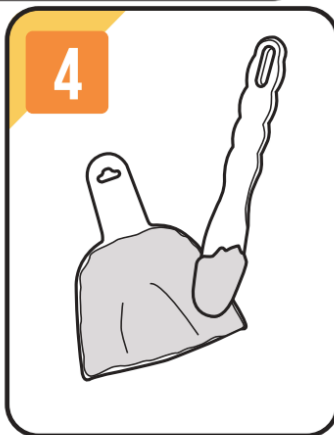
Spaulding's Classifications

maxill inc.


Canadian Head Office
80 Elm Street
St. Thomas, ON N5R 6C8
Toll Free: 1-800-268-8633
Phone: 1-519-631-7370
Fax: 519-631-3388


USA Head Office
500 West Main Street
Cortland, OH 44410
Toll Free: 1-855-462-9455
Phone: 1-330-637-1176
Fax: 1-330-637-1154

	20 mL water (≈1 scoop)	+	9 g maxsèt (≈1 scoop)	1 
	40 mL water (≈2 scoops)	+	18 g maxsèt (≈2 scoops)	2 



INITIAL SETTING TIME* **1m 35s**

6 


ONLY 30 s
*in the mouth**

(*)
Measurements
taken at 23°C
with deionised
water.



Spaulding's Classifications

maxill inc.

Canadian Head Office
80 Elm Street
St. Thomas, ON N5R 6C8
Toll Free: 1-800-268-8633
Phone: 1-519-631-7370
Fax: 519-631-3388

USA Head Office
500 West Main Street
Cortland, OH 44410
Toll Free: 1-855-462-9455
Phone: 1-330-637-1176
Fax: 1-330-637-1154