

maxill chrömax



**SPEARMINT
FLAVOUR**
Item 53775

TYPE I: Fast setting compliant with ISO 21563 specification.

maxill chrömax does not require special treatment before use.

PURPOSE: maxill chrömax is a chromatic alginate with long dimensional stability specifically studied for all those applications where a longer working time is essential.

maxill chrömax alginate can be used for the preparation of:

- Preliminary models
- Study models
- Mobile prosthesis
- Removable and fixed prosthesis antagonist

INSTRUCTIONS:

Follow the powder/water ratio. Pour the powder into the water, and mix vigorously. Work in compliance with the declared times and chromatic phases. Measurements were taken at 23°C with deionized water. When using specific mechanical mixers for alginate-based impression materials, follow the manufacturer's instructions. Variations in powder/water ratio, temperature and/or water hardness, and speed and/or friction during mixing can cause changes in the working time and the initial setting time.

The colour changes indicate the different work phases.

The chromatic phase indicator helps with the preparation of impressions without having to worry about temperature changes or specific water hardness. Temperature changes modify working and setting time and the colour indicator will change accordingly.

The chromatic phase indicator is a guarantee for perfect mixing and timely insertion in the oral cavity.

MIX	LOAD	INSERT
After mixing water and powder, the compound colour turns to purple, it is time to mix.	Pink indicates it is time to load the compound into the impression tray.	Blue indicates it is time to insert the loaded tray into the oral cavity.

IMPRESSION CARE AND STORAGE:

Wash, rinse, and disinfect the impression (specific alcohol-based spray disinfectants are recommended). By keeping the impression in a sealed plastic bag, without adding anything else, the model can be poured within 168 hours. **Do not leave the impression in open air or immersed in water.**

WARNINGS: For use, wear face mask, protective gloves, eye protection, and protective clothing. The powder is "Dust Free", however, caution is recommended to avoid inhalation. In case of contact with eyes wash thoroughly with water. After using the powder, close the bag immediately. Store in a cool and dry place. Use within the expiry date. Do not use on patients allergic to mint flavour.




Spaulding's Classifications

maxill inc.

Canadian Head Office
80 Elm Street
St. Thomas, ON N5R 6C8
Toll Free: 1-800-268-8633
Phone: 1-519-631-7370
Fax: 519-631-3388


USA Head Office
500 West Main Street
Cortland, OH 44410
Toll Free: 1-855-462-9455
Phone: 1-330-637-1176
Fax: 1-330-637-1154



20 mL water
(≈1 scoop)

+

9 g chrömax
(≈1 scoop)

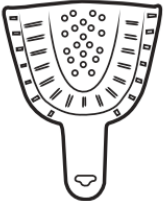

40 mL water
(≈2 scoops)

+

18 g chrömax
(≈2 scoops)

1




2




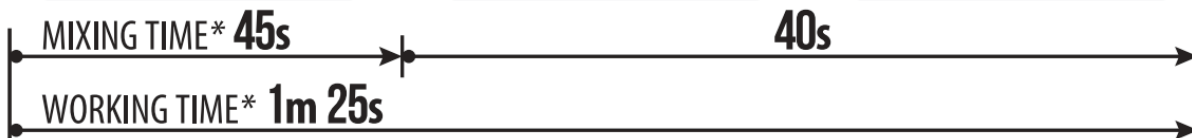
3



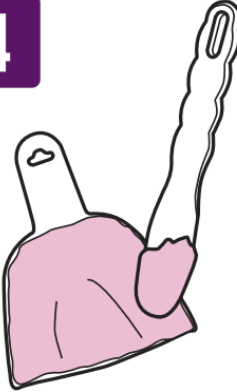

4



5

6

ONLY 30 s
in the mouth*

(*)
Measurements
taken at 23°C
with deionised
water.

