## STARTING WITH A NEW RIBBON HOLDER







Insert the *Ribbon Holder* onto cartridge and turn. Pull the *Ribbon Holder* off the cartridge. The *Transfer Connector* will now be detached from the *Ribbon Holder*. Proceed with steps below.

## STARTING WITH A **USED RIBBON HOLDER**



For best results - Remove air bubbles from each cartridge before transferring materials.



Place donor cartridge in the gun and insert a *Transfer Connector* into the tip of the cartridge.



Insert the *Ribbon Holder* onto the cartridge, using the arrow indicators to properly align as shown.



Connect the recipient cartridge, ensuring that the catalyst and base positions match up.



Rotate the *Ribbon Holder* to lock the two cartridges together. Transfer the remaining material to the recipient cartridge.

Reverse the rotation of the *Ribbon Holder*to unlock the cartridges. Remove the empty cartridge
and dispose of it with the *Transfer Connector*.
The *Ribbon Holder* can be reused.



