


# Cleaning Instructions for **maxill Mouth Props**

(Item 41515, 41516, 41517)

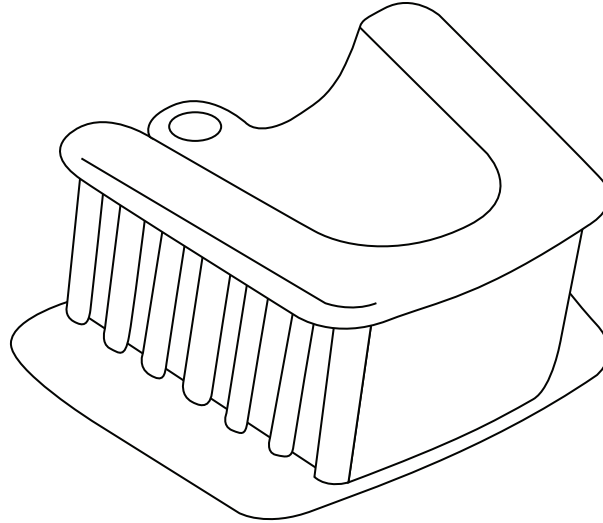

## **Cleaning**

maxill mouth props are to be cleaned with an enzymatic cleaner via ultrasonic bath or instrument washer. After cleaning rinse the mouth prop thoroughly to ensure full removal of any enzymatic cleaning residue. Inspect to ensure all gross debris has been removed.

## **Sterilization**

Ensure the maxill mouth prop has been cleaned properly. Reinspect to confirm it is free of debris. Package in a suitable size sterilization pouch and/or compartment in a cassette that does not come into contact with other instruments. Sterilize the mouth prop in a steam sterilizer up to 134°C (273°F).

**NOTE:** Do not disinfect the mouth prop as this is not an acceptable form of reprocessing. The mouth prop is a semi-critical item as per Spaulding's Classification as it comes into contact with oral mucosa.