

Plaster & Stone Remover

• Cleaner & Remover

Product Description

Plaster & Stone Remover (PSR) cleans away plaster and stone from dentures, partials, bridges, flasks, articulators & orthodontic appliances and is effective in the removal of permanent cement.

Ready-to-Use (RTU):

PSR ready-to-use; no dilution is required. Dispense directly from the bottle.

Directions:

Pour an adequate amount of PSR into a glass beaker suitable to a level that will totally immerse the items to be cleaned. Ensure the beaker is covered. Place beaker into ultrasonic unit with the beaker sitting in the carrier bath at a depth of about 3 cm. Do not use PSR directly in the ultrasonic tank. Ensure the carrier bath will not enter the beaker. Sonic clean for 5 to 10 minutes. Then thoroughly rinse items with water.

Please read the MSDS for safe handling information.

Physical Data:

Clear, amber liquid
pH: approximately 12.90

Product Availability:

900 mL Bottle#60012
4 L Jug.....#60011



The information provided is given in good faith and believed to be accurate, but no warranty or guarantee is given or implied. Conditions, applied surfaces, instruments /devices, application temperatures, application times and targeted residues under which the product is being applied will vary from the conditions of normal laboratory efficacy testing. Therefore, each health care facility should conduct its own efficacy testing in order to verify the efficacy of the product under its own normal operating conditions to meet the specific requirements of the workplace.