

# Tartar & Stain Remover

## • Cleaner & Remover

### Product Description

Tartar & Stain Remover cleans away food and tobacco stains and calculus from dentures, bridges & orthodontic appliances and removes rust, flux, solder, calcium hydroxide and hard water scale.

### Ready-to-Use (RTU):

TSR ready-to-use; no dilution is required. Dispense directly from the bottle.

### Directions:

Pour and adequate amount of TSR into a glass beaker suitable to a level that will totally immerse the items to be cleaned. Ensure the beaker is covered. Place beaker into ultrasonic unit with the beaker sitting in the carrier bath at a depth of about 3 cm. Do not use TSR directly in the ultrasonic tank. Ensure the carrier bath will not enter the beaker. Sonic clean for 5 to 10 minutes. Then thoroughly rinse items with water.

Mouth items, such as dentures should be neutralized in a solution of 1 teaspoon of sodium bicarbonate diluted in a cup of water. Rinse the Item again.

Please read the MSDS for safe handling information.

### Physical Data:

White, hazy liquid; sulphur odour

pH: approximately 1.30

### Product Availability:

900 mL Bottle .....#60016  
 4 L Jug .....#60010



The information provided is given in good faith and believed to be accurate, but no warranty or guarantee is given or implied. Conditions, applied surfaces, instruments /devices, application temperatures, application times and targeted residues under which the product is being applied will vary from the conditions of normal laboratory efficacy testing. Therefore, each health care facility should conduct its own efficacy testing in order to verify the efficacy of the product under its own normal operating conditions to meet the specific requirements of the workplace.