

Cleanse-a-dent™ • Cleaner

Product Information

Overnight soaking is a thing of the past! maxill brand Cleanse-a-dent is a powerful, fast acting eco-friendly denture cleaner. Cleanse-a-dent will remove soil and light stains from dentures in minutes.

Economical Concentrate:

Cleanse-a-dent is a concentrate to be diluted at 1:5 (approximately 45 mL into 225 mL of water).

4 L makes up to 24 L.

Directions:

To remove loose debris, prerinse dentures in warm running water. To make 1:5 dilution add 45 mL of Cleanse-a-dent to the maxill M-LYD denture bath and fill with lukewarm tap water to the horizontal mark. Allow dentures to soak for 10 minutes. Afterwards brush the dentures under lukewarm running, water.

For stubborn stains, soak the dentures for a longer period of time. Always rinse the dentures very thoroughly before inserting them into the mouth.

Prepare a fresh solution when solution becomes visibly dirty.

Please read the MSDS for safe handling information.

Physical Data:

Clear, colourless liquid with no odour
pH: approximately 11.90

Product Availability:

425 mL Bottle #61502
4 L Jug #61500



The information provided is given in good faith and believed to be accurate, but no warranty or guarantee is given or implied. Conditions, applied surfaces, instruments /devices, application temperatures, application times and targeted residues under which the product is being applied will vary from the conditions of normal laboratory efficacy testing. Therefore, each health care facility should conduct its own efficacy testing in order to verify the efficacy of the product under its own normal operating conditions to meet the specific requirements of the workplace.