

Zymax[™] 128

Enzymatic Cleaning Solution

#60035
1 L Squeeze 'n'
Measure Bottle



Zymax 128 is a versatile detergent and is compatible with many materials including plastics, rubber, stainless steel, and fabrics that are soiled with organic debris. If in doubt, first test for compatibility. Working solutions can be used as instrument soaks, in ultrasonic units and in evacuation systems. Consult equipment manuals before using.

Directions: Dilute 1-part Zymax 128 with 127 parts water, near room temperature. E.g. Mix 15 mL / ½ oz. of Zymax 128 with water making up a working solution to 1.9 L / 2 qt. Do not add any other chemicals. The solution should be changed daily or when heavy use affects its performance.

Manual Cleaning & Instrument Soak: Soak instruments for 5 to 10 minutes in the working solution. Additional time may be necessary for dried blood and other protein debris. Do not leave equipment in the solution for more than 30 minutes. Immerse instruments completely and aspirate solution into any channels and crevices. Following the soak, visually inspect for cleanliness and rinse thoroughly with warm water. Dry instruments prior to disinfection or sterilization.

WARNING

Causes mild skin irritation
Causes eye irritation

Wash hands thoroughly after handling.
Do not touch eyes.

If skin irritation occurs: Get medical help.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

If eye irritation persists: Get medical help.



SPALDING'S
CLASSIFICATIONS

maxill inc.

Canadian Head Office

80 Elm Street, St. Thomas, ON N5R 6C8

Toll Free: 1-800-268-8633

Phone: 1-519-631-7370 Fax: 1-519-631-3388

USA Head Office

500 West Main Street, Cortland, OH 44410

Toll Free: 1-855-462-9455

Phone: 1-330-637-1176 Fax: 1-330-637-1154