

Zymax^{LF}

Low Foaming Enzymatic Cleaner

#60034
1 L Squeeze 'n'
Measure Bottle



DIRECTIONS:

Shake bottle before each use. To dispense, loosen the cap on the measuring chamber (foil seal removed). Then gently squeeze the bottle to add liquid into the measuring chamber to the desired amount. Remove the cap and pour out. Keep caps snugly closed on the bottle when not in use.

Dilute 1 part Zymax LF with 127 parts water, near room temperature. E.g. Mix 15 mL / ½ fl.oz. of Zymax LF with water making up a working solution to 1.9 L / 2 qt. Do not add any other chemicals. The working solution should be changed daily or when heavy use affects its performance.

DIRECTIONS FOR CLEANING INSTRUMENTS:

Soak instruments for a minimum of 2 minutes. Longer soaking time may be required for tough or dried matter. Flush solution through channels to ensure thorough cleaning. May be used in an ultrasonic unit. Rinse instruments by aspirating water through all channels and hinges. Completely rinse solution from instruments. Dry instruments before proceeding to the disinfection process. Follow manufacturer's directions for sterilization. Discard detergent after each use or when visibly contaminated.

DIRECTIONS FOR CLEANING EVACUATION EQUIPMENT:

1. Prepare 1.9 L / ½ gallon of a working solution.
2. Circulate via evacuator hose for 90 seconds to flush the system of organic waste and odors, from point of intake to collector traps.
3. Repeated daily cleaning may be advisable.

WARNING

Causes mild skin irritation

Causes eye irritation

Wash hands and exposed areas thoroughly after handling. Do not touch eyes.

If skin irritation occurs: Get medical help.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical help.

Store at room temperature. Keep out of reach of children.



SPALDING'S
CLASSIFICATIONS

maxill inc.

Canadian Head Office

80 Elm Street, St. Thomas, ON N5R 6C8

Toll Free: 1-800-268-8633

Phone: 1-519-631-7370 Fax: 1-519-631-3388

USA Head Office

500 West Main Street, Cortland, OH 44410

Toll Free: 1-855-462-9455

Phone: 1-330-637-1176 Fax: 1-330-637-1154