

maxill

ITEM 60037

UCS³³

ULTRASONIC CLEANING SOLUTION

1 L • 33.8 fl oz (1.06 qt)



Lemon Scent

33_x Concentrate

Directions for use: Use the working solution directly in the ultrasonic unit. Add 1 part UCS³³ to 32 parts water. (e.g. 30 mL UCS³³ to 970 mL of water or 1 fl oz of UCS³³ to 32 fl oz of water). Subject instruments to ultrasonic cleaning for 3 to 5 minutes. Rinse instruments well with water and air dry. For best results, change your ultrasonic bath daily or earlier if solution becomes visibly dirty.

Prevention: Observe good industrial hygiene practices.

Response: Wash hands after handling.

Storage: Store away from incompatible materials.

Disposal: Dispose of waste and residues in accordance with local authority requirements.

REF 60037

REV.2018/05



0 60705 60037 0

Made in Canada with domestic and imported parts.

maxill
1-800-268-8633
www.maxill.com

Manufactured for: **USA: maxill inc.**
Cortland, OH, USA, 44410

Manufacturer: **Canada: maxill inc.**
St. Thomas, ON, CA, N5R 6C8