maxill

Plaster & Stone Remover

Cleans away plaster and stone from dentures, partials, bridges, flasks, articulators & orthodontic appliances.

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#60011

REV.2025/06

Directions: PSR is ready-to-use. Pour an adequate amount of PSR into a glass beaker suitable to a level that will totally immerse the items to be cleaned. Ensure glass beaker is covered. Place glass beaker into ultrasonic unit with at least 3 cm of the beaker sitting in the carrier bath. Do not use PSR directly in the ultrasonic tank. Ensure carrier bath will not enter beaker. Sonic clean for 5 to 10 minutes. Then thoroughly rinse items with water. Change solution after every use to avoid cross contamination. DO NOT USE ON ALUMINIUM!

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Plaster and Stone Remover

DANGER

Harmful if swallowed.
Causes skin irritation.
Causes serious eye damage.

PREVENTION

Wash hands thoroughly after handling. Do not touch eyes. Do not eat, drink or smoke when using this product. Wear protective gloves. Wear eye protection.

RESPONSE

IF SWALLOWED: Get medical help. Rinse mouth.

IF ON SKIN: Wash with plenty of water. Specific treatment: (see First Aid on the SDS). If skin irritation occurs: Get medical help. Take off contaminated clothing and wash it before reuse.

IF IN EYES: Immediately rinse with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Get medical help.

DISPOSAL

Dispose of contents in accordant with all regulations that apply to your region.

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maxill inc.
www.maxill.com 1-800-268-8633

Manufacturer:
maxill inc.
St. Thomas, ON
N5R 6C8 Canada

Distributed in USA by:
maxill inc.
Cortland, OH
44410 USA

REV.2025/06