**Directions:** Tartar & Stain Remover (TSR) is ready-to-use. Pour an adequate amount of TSR into a glass beaker suitable to a level that will totally immerse the items to be cleaned. Cover beaker and place it into an ultrasonic unit with 3 cm of the beaker sitting in the carrier bath. Ensure carrier bath will not enter the beaker. Sonic clean for 5 to 10 minutes. Immediately thoroughly rinse items with water. Mouth items, such as dentures, etc., should be neutralized in a solution of 1 teaspoon of sodium bicarbonate diluted in a cup of water. Rinse item again. DO NOT USE ON ALUMINIUM!

## Tartar and Stain Remover

### DANGER

Causes severe skin burns and eye damage.  
May be harmful if swallowed.  
May be harmful if inhaled.  
May be corrosive to metals

### PREVENTION

Wash hands thoroughly after handling.  
Wear protective gloves and eye protection. Do not breathe mist.

### RESPONSE

**IF SWALLOWED:** Rinse mouth. Do NOT induce vomiting. Specific treatment: give large amounts of milk or water. Immediately call a POISON CENTER or physician.

**IF ON SKIN (or hair):** Remove immediately all contaminated clothing. Rinse skin with water. Wash contaminated clothing before reuse. Immediately call a POISON CENTER or physician.

**IF INHALED:** Remove victim to fresh air and keep at rest in a position comfortable for breathing. Immediately call a POISON CENTER or physician.

**IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a POISON CENTER or physician.

Absorb spillage to prevent material damage.

### STORAGE

Store locked up. Store only in original container.

### DISPOSAL

Dispose of contents/container in accordance with local, provincial/state and national regulations.

Made in Canada with domestic and imported parts.

Distr. by/par

**maxill inc.**

Cortland, OH, USA 44410  
St. Thomas, ON, Canada N5R 6C8

1-800-268-8633  
www.maxill.com

**maxill**

# Tartar & Stain Remover

Cleans away food & tobacco stains and calculus from dentures, bridges & orthodontic appliances and removes rust, calcium hydroxide and hard water scale.

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