

## Instructions For Use

# maxill Bib Clipz

Autoclavable Silicone Bib Clips (2 clips per bag)

#54201 - Navy Blue #54202 - Light Blue #54203 - Green #54204 - Lavender



SPAULDING CLASSIFICATION: Non-Critical Item; Contact intact skin but not mucous membranes. Requires low-level disinfection.

> NOTE: A reusable non-critical item has the choice of either disinfection or steam sterilization. At minimum a non-critical item must be disinfected.

**INTENDED USE:** For use by dental professionals in the recommended indications.

**MATERIALS:** Medical grade silicone, Stainless steel.

CLEANING: Reusable non-critical patient care items must be disinfected after use. Clean and disinfect thoroughly between each use with an approved Health Canada or EPA-registered disinfectant.

STEAM STERILIZATION: Autoclavable in Class B, pre-vacuum steam sterilizer up to 275°F (135 °C) for 4 mins.

#### **IMPORTANT NOTICE**

As with any stainless steel component, corrosion is a possibility. It is the responsibility of the healthcare professional to inspect any instrument or device of stainless steel material for degradation (i.e. rust, corrosion, pitting). If the integrity of the instrument or device is compromised, cease use.

### Contraindications

- Cleaning agents with chlorine or chlorine as the active ingredient are corrosive to stainless steel and should not be used.
- Do not use cold sterilization methods (chemical process) for the sterilization of the bib blipz. These chemical agents can cause corrosion of the product.



#### maxill inc.

Canadian Head Office 80 Elm Street

St. Thomas, ON N5R 6C8 Toll Free: 1-800-268-8633 Phone: 1-519-631-7370 Fax: 519-631-3388

**USA Head Office** 500 West Main Street Cortland, OH 44410 Toll Free: 1-855-462-9455 Phone: 1-330-637-1176 Fax: 1-330-637-1154

Rev.2: 2024-02-02 MP-REF-54201D