


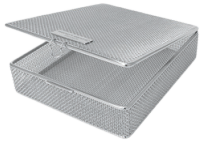


## Mesh Baskets

### Hinged Lid with Snap Closure

#SCACC001- 156mm x 216mm x 54mm  
#SCACC002- 216mm x 178mm x 51mm



#SCACC005- 40mm x 40mm x 20mm  
#SCACC006- 80mm x 40mm x 20mm  
#SCACC007- 160mm x 40mm x 20mm  
#SCACC008- 70mm x 70mm x 30mm  
#SCACC009- 90mm x 90mm x 50mm  
#SCACC010- 80mm x 40mm x 40mm

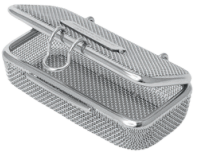

**Ohio Forge Mesh Baskets** are designed to hold instruments with smaller parts that are difficult to contain during cleaning procedures or transportation. Ohio Forge Mesh Baskets can be used in ultrasonic baths and automatic washers.

**Material:** 304 stainless steel ultra-fine woven mesh

#### CLEANING NOTE:

Although these are not used in direct client care, mesh baskets require care as follows:

- If used in an ultrasonic bath, include mesh baskets as part of your daily disinfection protocols
- If used in automatic washer-disinfector, include mesh basket as part of the cleaning cycle
- If there is visible soil buildup on mesh baskets, debris with running water and brush to remove debris. Clean between mesh with small stainless steel/nylon bristle cleaning brush if noticeable debris present.

#### STORAGE:

If used in washer-disinfector you may store inside the unit to use next day. If used with an ultrasonic cleaning bath, disinfect with closing protocols before storing away in a cupboard or in ultrasonic bath.

Do not store in open area even if disinfection protocols have been executed.

#### STERILIZATION:

Although these are not recommended for sterilization, in the event you no longer use daily, Ohio Forge mesh baskets may be sterilized prior to long term storage.

- Sterilize according to AAME/ANSI ST55 and AAMI/ANSI ST8.
- Validate cycle according to AAME/AAMI ST 79 (valid IQ/OQ (commissioning) and product specific performance qualification (PQ)).
- It is recommended that temperatures do not exceed 360°F (182°C) during any sterilization process.



Spaulding's Classifications

#### maxill inc.

Canadian Head Office  
80 Elm Street  
St. Thomas, ON N5R 6C8  
Toll Free: 1-800-268-8633  
Phone: 1-519-631-7370  
Fax: 519-631-3388

USA Head Office  
500 West Main Street  
Cortland, OH 44410  
Toll Free: 1-855-462-9455  
Phone: 1-330-637-1176  
Fax: 1-330-637-1154