



Steri Container with Mat



#SCHP0002 - 80mm x 60mm x 25mm
 #SCHP0003 - 100mm x 60mm x 25mm
 #SCHP0004 - 150mm x 40mm x 30mm



Ohio Forge Steri Containers are manufactured for use in any cleaning and sterilization process. This item can be used in ultrasonic baths, automatic washers, and autoclaves.

Material: 304 stainless steel and medical-grade silicone.

CLEANING

- After use, wipe and clean the instrument of visible debris before placing in the Ohio Forge Steri Container
- Clean the container and contained instruments in enzymatic cleaner or place in automatic washer-disinfector using manufacturers recommended solution for automatic unit.
- Ohio Forge Steri Containers are fitted with silicone mat. The mat is also suitable for use with any decontamination, cleaning and sterilization process.
- It is important to ensure complete drying before packaging and sterilization.

STERILIZATION

- Sterilize according to AAME/ANSI ST55 and AAMI/ANSI ST8.
- Validate cycle according to AAME/AAMI ST 79 (valid IQ/OQ (commissioning) and product specific performance qualification (PQ)).
- Ensure item is free of debris after the cleaning process.
- Package for sterilization using appropriate size sterilization pouches or surgical wrap.
- Ensure package does not touch other items when placing in sterilizer chamber.
- It is recommended that temperatures do not exceed 360°F (182°C) during any sterilization process.
- It is recommended to follow the manufacturers instructions for reprocessing for the instruments held within in the container.
- Allow to dry completely before removing from sterilizer chamber.



Spaulding's Classifications

maxill inc.

Canadian Head Office
 80 Elm Street
 St. Thomas, ON N5R 6C8
 Toll Free: 1-800-268-8633
 Phone: 1-519-631-7370
 Fax: 519-631-3388

USA Head Office
 500 West Main Street
 Cortland, OH 44410
 Toll Free: 1-855-462-9455
 Phone: 1-330-637-1176
 Fax: 1-330-637-1154

Rev.0: 2024-09-11

MP-REF-SCHP0002D