



#SMIK0001- Autoclavable Suction Mirror- Intro Kit
 #SMIK0004- Suction Mirror LV Adaptor
 #SMIK0005- Suction Mirror HV Adaptor
 #SMIK0006- Suction Mirror Handle- Single Red
 #SMIK0007- Suction Mirror Handle- Single Green
 #SMIK0008- Suction Mirror Handle- Single Blue
 #SMIK0009- Suction Mirror Handle- Single Yellow
 #SMIK0010- Suction Mirror Handle- Single Grey
 #SMIK0011- Suction Mirror- Hose
 #SMIK0012B- Suction Mirror Head- Single #4
 #SMIK0013- Suction Mirror Cleaning Brush

SPAULDING CLASSIFICATION: Semi-Critical; Comes into contact with mucous membranes or non-intact skin.

INTENDED USE: For use by dental professionals in the recommended indications.



NON-STERILE: STERILIZE BEFORE USE



CLEANING AND STERILIZATION:

- Disassemble all components for cleaning and sterilization
- Clean and reprocess as per protocols pertaining to lumens according to regulatory bodies of your geographical area.
- Manually clean mirror handle, mirror head, and adaptors with a soft brush (included) to remove any debris or buildup
- Washer-disinfector may be used. Refer to Washer-disinfector manual for recommended cycle type for lumens
- **Mirror head and mirror handle must be reprocessed after each use.**

Only use Instrument Washer certified by FDA for medical/dental use, using a neutral pH cleaner certified for use in Medical Dental Instrument Washers.

Steam sterilization: 132°C (270°F) for 4 minutes and a minimum of 25 minutes drying time. Do not exceed 135°C (275°F).

Gravity displacement: 121°C (250°F) for 30 mins with 15-20 minutes of drying time. Do not exceed 125°C (257°F).

Hose Cleaning and Maintenance: If reprocessing after each use cannot be completed, follow your local standards for cleaning and disinfecting evacuation lines according to regulatory bodies of your geographical area.

- Clean hose with evacuation cleaner (prepared according to the manufacturer's directions for use) between each use/patient.
- Remove any visible soil and thoroughly wipe down entire surface of hose and adaptor between each use with an approved Health Canada or EPA-registered disinfectant.



Spaulding's Classifications

maxill inc.

Canadian Head Office
 80 Elm Street
 St. Thomas, ON N5R 6C8
 Toll Free: 1-800-268-8633
 Phone: 1-519-631-7370
 Fax: 519-631-3388


USA Head Office
 500 West Main Street
 Cortland, OH 44410
 Toll Free: 1-855-462-9455
 Phone: 1-330-637-1176
 Fax: 1-330-637-1154



#SMIK0001- Autoclavable Suction Mirror- Intro Kit
#SMIK0004- Suction Mirror LV Adaptor
#SMIK0005- Suction Mirror HV Adaptor
#SMIK0006- Suction Mirror Handle- Single Red
#SMIK0007- Suction Mirror Handle- Single Green
#SMIK0008- Suction Mirror Handle- Single Blue
#SMIK0009- Suction Mirror Handle- Single Yellow
#SMIK0010- Suction Mirror Handle- Single Grey
#SMIK0011- Suction Mirror- Hose
#SMIK0012B- Suction Mirror Head- Single #4
#SMIK0013- Suction Mirror Cleaning Brush

INSTRUCTIONS TO ASSEMBLE:

1. Insert chosen adaptor into suction holder.

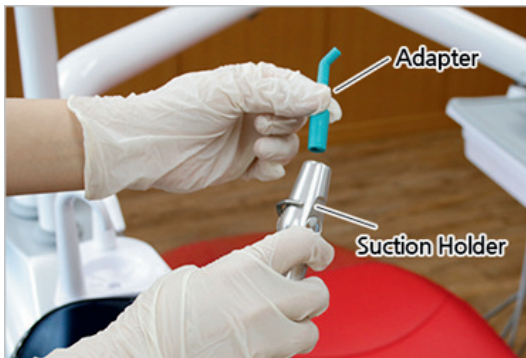

2. Connect hose to adaptor.



3. Insert mirror suction handle into hose adaptor, push firmly to avoid dislodging.



DISASSEMBLE ALL COMPONENTS PRIOR TO CLEANING AND STERILIZATION:



Cleaning Brush: For cleaning of mirror handle, mirror head stem, mirror head suction holes, adaptors, and hose.



Spaulding's Classifications

maxill inc.

Canadian Head Office
80 Elm Street
St. Thomas, ON N5R 6C8
Toll Free: 1-800-268-8633
Phone: 1-519-631-7370
Fax: 519-631-3388

USA Head Office
500 West Main Street
Cortland, OH 44410
Toll Free: 1-855-462-9455
Phone: 1-330-637-1176
Fax: 1-330-637-1154