Multi-enzyme - A proprietary blend of enzymes lift and carry away organic material including blood, tissue, and feces

Safe to Use - Penetrates hard-to-reach places reducing the need for intensive manual cleaning of contaminated surfaces

pH Neutral - Will not rust or damage instruments

Directions for cleaning instruments:

1. Shake bottle before each use.
2. Saturate instruments and equipment with spray foam as soon as possible after the procedure.
3. Keep instruments and equipment wet with product until the decontamination process is started.
4. Rinse with potable water.
5. Follow manufacturer's directions for instrument sterilization.

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maxill

Zymax QS

QUICK SHOT ENZYMATIC CLEANING FOAM MOUSSE NETTOYANTE ENZYMATIQUE

RTU Multi-Enzyme Detergent Spray Foam pH Neutral Detergent

Zymax QS is a RTU multi-enzyme detergent formulated to be especially effective in the removal of organic material such as blood, tissue, mucus, feces, and other organic material found on instruments.

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946 mL (32 FL OZ)

Zymax QS QUICK SHOT ENZYMATIC CLEANING FOAM

WARNING

Causes mild skin irritation
Causes eye irritation
Wash hands and exposed areas thoroughly after handling.
If skin irritation occurs: Get medical help.
IF IN EYES: Rinse cautiously with water for several minutes.
Remove contact lenses, if present and easy to do. Continue rinsing.
If eye irritation persists: Get medical help.

Store at room temperature. Keep out of reach of children.

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Made in Canada with domestic and imported ingredients

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REF 607056002501

maxill inc.
www.maxill.com 1-800-268-8633

Manufacturer:
maxill inc.
St. Thomas, ON
N5R 6C8 Canada

Distributed in USA by:
maxill inc.
Cortland, OH
44410 USA

+D892607056002501

REV 2026/05