maxill

DIN 02246644

Unscented/Sans parfum

#60006U

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Ethanol	79.0% w/w
Benzalkonium Chloride	0.2% w/w
o-Phenylphenol	0.1% w/w

**INSTITUTIONAL USE ONLY
READ THE LABEL BEFORE USING**

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tb Minuteman is an intermediate level hard surface disinfectant as well as a tuberculocide, virucide and fungicide. tb Minuteman is designed for use in dental and medical clinics, hospitals and nursing homes where disinfection and prevention of cross-contamination is of prime importance.

tb Minuteman is intended for use on pre-cleaned, hard non-porous, non-food contact inanimate medical equipment surfaces such as bed & security rails, commodes, miscellaneous medical equipment surfaces & knobs, EMS equipment, examination tables, and any other equipment surfaces that are potentially contaminated with disease-causing microorganisms. tb Minuteman will not stain or leave residual smears after drying. tb Minuteman is not for use in hospital nurseries.

If you are uncertain as to the material composition of your surface, confirm compatibility with the manufacturer before proceeding. For plastic and painted surfaces, spot test on an inconspicuous area before use.

ORGANISMS CONTROLLED / CONTACT TIMES: When used as directed on pre-cleaned hard, non-porous, non-food contact surfaces, tb Minuteman is a disinfectant, effective against the following microorganisms.

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ANGER

PRECAUTIONARY STATEMENTS:

Extremely flammable liquid and vapor. Causes skin irritation. Causes serious eye irritation. May cause respiratory irritation. May cause drowsiness or dizziness. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Keep container tightly closed. Take action to prevent static discharges. Use explosion-proof ventilating equipment. Avoid breathing mist or vapor. Use only outdoors or in a well-ventilated area. Wash hands thoroughly after handling. Do not touch eyes. Wear protective gloves and eye protection. In case of fire: Use appropriate media to extinguish.

FIRST AID:

IF ON SKIN (or hair): Take off immediately all contaminated clothing. Rinse skin with water. If skin irritation occurs: Get medical help. Take off contaminated clothing and wash it before reuse. Specific treatment: see further information on this label.
IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical help.
IF INHALED: Remove person to fresh air and keep comfortable for breathing. Get medical help you feel unwell.

STORAGE: Store in a well-ventilated place. Keep container tightly closed. Store locked up. Keep cool.

DISPOSAL: Recycle this container or discard container in trash. For information on disposal of unused, unwanted product, contact the manufacturer or the appropriate municipal/provincial/territorial agency or product stewardship organization.

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DIRECTIONS FOR USE

This product is not to be used as a terminal sterilant / high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization/high-level disinfection.

tb Minuteman is a Ready to Use product (RTU) and should be used at full strength. Pre-clean the surface by liberally applying tb Minuteman then wiping with a clean cloth to remove organic soil. To disinfect the cleaned surface, apply tb Minuteman liberally, until the surface is thoroughly wet. Ensure the surface remains wet for at least 1 minute at Room Temperature. Then wipe the surface dry.

*This product effectively kills Human Immunodeficiency virus 1 (HIV-1) in settings where the virus would be expected to be encountered such as settings where contamination by blood or body fluid is likely.

Special Decontamination Procedures:

Personal protection: Personnel that clean items soiled with blood or body fluids must wear appropriate personal protective equipment (i.e. barrier protection), such as disposable gloves, gowns, masks or eye coverings;

Cleaning: Target surfaces must be cleaned prior to disinfection, as the presence of heavy soil on a surface may reduce the intended disinfectant efficacy of the product; and

Infectious materials disposal: Any materials used in the cleaning process that may contain blood or body fluids are to be disposed of immediately in accordance with local regulations for infectious materials disposal.

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